

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.
DATE 1/12/2015 TIME 1926
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO - BLACK CANYON

- (67) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
~~Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$~~
- (87) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(61) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(64) 4. Air blank completed.
(64) 5. Calibration check completed. Test results 0, 100 AC.
(61) 6. Air blank completed.
(64) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Moody 7486

DPS Form Exh G-2 (Rev 05-01)

STANDARD CHANGE:
LOT: 25414 100 A4
CYL: 033
EXP: 10/5/2016

DATE / TIME COLLECTED



7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62651-0790
217-245-2183 • Fax: 217-243-7634 • www.ilmoproducts.com

Certificate of Analysis

Certificate ID: 7177
Part #: BAC105L100T
Cylinder Size: 105L
Lot Number: 25414100A4
Expiration: 10/5/2016


0.100 BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents: 105 Liters @ 1000 psig 70°F (21°C)

Component:	Concentration:	Accuracy:	Method:
Ethanol	260 ppm	+/- 0.002 or 2%	NDIR
Nitrogen	Balance	BAC whichever is greater	

*NIST Standard Reference Material
Cylinder No. CCI4290 / Job No. 09160202
Certified 212.8 µmol/mol Ethanol in Nitrogen
for ILMO Products Co., Jacksonville, IL

Store in dry area, away from sources of heat, ignition
and direct sunlight. Do not allow storage area to
exceed 52 °C (125 °F).


Specialty Gas Lab Tech

10/09/14
Date



Distributed by: CMI Inc.
316 East Ninth Street
Owensboro, KY 42303
Phone 866-835-0690
www.alcoholtest.com

INTOXILYZER 8000
Location: YCSO - BLACK CANYON
Serial Number: 80-000289
Core Version: 8105.46
01/12/2015 18:26:04

Standard Lot#: 25414100A4
Last Changed By: L. MOODY #7486

OAS: L. MOODY #7486
YAUAPAI CSO

Operator: L. MOODY #7486
YAUAPAI CSO

Test	g/210L	Time
Air Blank	0.000	18:26:33
0.100 Cal Check	0.100	18:26:54
Air Blank	0.000	18:27:23

Preventative Maintenance
Performed -- Timer Reset