

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.
DATE 2/2/2015 TIME 1725
INTOXILYZER SERIAL # 80-000289 LOCATION 4650 - BLACK CANYON

- (67) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC. into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (67) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(67) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(67) 4. Air blank completed.
(67) 5. Calibration check completed. Test results 0. 101 AC.
(67) 6. Air blank completed.
(67) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Logan Moody

INTOXILYZER 8000
Location: YCSO - BLACK CANYON
Serial Number: 80-000289
Core Version: 8105.48
02/02/2015 17:25:01

Standard Lot#: 25414100A4
Last Changed By: L. MOODY #7486

QAS: L. MOODY #7486
YAUAPAI CSO

Operator: L. MOODY #7486
YAUAPAI CSO

Test	g/210L	Time
Air Blank	0.000	17:28:24
0.100 Cal Check	0.101	17:28:45
Air Blank	0.000	17:29:15

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.
DATE 2/2/2015 TIME 1729
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO - BLACK CANYON

(67) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (67) 1. Clock time check.
(67) 2. Date check.

OPERATIONAL TESTS

- (67) 1. Alcohol-free subject test result 0. 000 AC.
(67) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(147) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(67) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES 67 NO _____

COMMENTS

SIGNATURE Logan Moody

INTOXILYZER 8000
 Location: YCSO - BLACK CANYON
 Serial Number: 80-000289
 Core Version: 8105.48
 02/02/2015 17:29:50

Standard Lot#: 25414100A4
 Last Changed By: L. MOODY #7486

DAS: L. MOODY #7486
 YAUAPAI CSO

Operator: L. MOODY #7486
 YAUAPAI CSO

Subject: SQAP, TEST
 DOB: 01/01/2000
 Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:30:43
Diagnostic Test	Pass	17:31:20
Air Blank	0.000	17:31:48
0.100 Cal Check	0.100	17:32:08
Air Blank	0.000	17:32:38
Subject Test	0.000	17:33:20
Air Blank	ABT*	17:33:28
Air Blank	RF!*	17:33:36

*Sequence Aborted
 **RF! Detect

Not a Successfully
 Completed
 Test Sequence

INTOXILYZER 8000
 Location: YCSO - BLACK CANYON
 Serial Number: 80-000289
 Core Version: 8105.48
 02/02/2015 17:33:45

Standard Lot#: 25414100A4
 Last Changed By: L. MOODY #7486

DAS: L. MOODY #7486
 YAUAPAI CSO

Operator: L. MOODY #7486
 YAUAPAI CSO

Subject: SQAP, TEST
 DOB: 01/01/2000
 Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:34:39
Diagnostic Test	Pass	17:35:17
Air Blank	0.000	17:35:44
0.100 Cal Check	0.100	17:36:05
Air Blank	0.000	17:36:34
Subject Test	0.121*	17:39:42
Air Blank	0.000	17:40:13
Air Blank	0.000	17:40:41
Subject Test	XXX**	17:41:06
Air Blank	IPS***	17:41:14

*Deficient Sample
 **Invalid Sample
 Begin new deprivation period
 ***Improper Sample

Not a Successfully
 Completed
 Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOOS AGENCY GAUAPAT CO. S.O.
DATE 2/2/2015 TIME 1742
INTOXILYZER SERIAL # 980-000289 LOCATION YCSO - BLACK CANYON

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$~~
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. _____ AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE LMOOS 7486

DPS Form Exh G-2 (Rev 05-01)

"UNSTABLE SIGNAL" ERROR. TEST SEQUENCE ABORTED.

INTOXILYZER 8000
Location: YCSO - BLACK CANYON
Serial Number: 80-000289
Core Version: 8105.48
02/02/2015 17:42:55

Standard Lot#: 25414100A4
Last Changed By: L. MOODY #7486

QAS: L. MOODY #7486
YAUAPAI CSO

Operator: L. MOODY #7486
YAUAPAI CSO

Test	g/210L	Time
Air Blank	0.000	17:43:24
Reference	UNS*	17:43:32
Air Blank	0.000	17:44:00

*Unstable Signal

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.

DATE 2/2/2015 TIME 1744

INTOXILYZER SERIAL # 80-000289 LOCATION YCSO - BLACK CANYON

- (M) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$~~
- (M) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (M) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (M) 4. Air blank completed.
- (M) 5. Calibration check completed. Test results 0. 100 AC.
- (M) 6. Air blank completed.
- (M) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Logan Moody

INTOXILYZER 8000
Location: YCSO - BLACK CANYON
Serial Number: 80-000289
Core Version: 8105.48
02/02/2015 17:44:16

Standard Lot#: 25414100A4
Last Changed By: L. MOODY #7486

OAS: L. MOODY #7486
YAUAPAI CSO

Operator: L. MOODY #7486
YAUAPAI CSO

Test	g/210L	Time
Air Blank	0.000	17:44:45
0.100 Cal Check	0.100	17:45:06
Air Blank	0.000	17:45:36

Preventative Maintenance
Performed -- Timer Reset