

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST KELLY WOHLWEND AGENCY YCSO

DATE 03/30/15 TIME 5:53 HOURS

INTOXILYZER SERIAL # 80-000789 LOCATION YCSO BLACK CANYON

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
 4. Air blank completed.  
 5. Calibration check completed. Test results 0.100 AC.  
 6. Air blank completed.  
 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Kelly Wohlwend 9811

INTOXILYZER 8000  
Location: YCSC - BLACK CANYON  
Serial Number: 80-000289  
Core Version: 8105.48  
03/30/2015 21:53:09

-----  
Standard Lot#: 25414100A4  
Last Changed By: L. MOODY #7486  
-----

\*\*\*  
GAS: K. WOHLWEND #9811  
YAUAPAI CO SO

-----  
Operator: K. WOHLWEND #9811  
YAUAPAI CO SO  
-----

Test	g/210L	Time
Air Blank	0.000	21:53:49
0.100 Cal Check	0.100	21:54:10
Air Blank	0.000	21:54:39

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST KELLY WOHLWEND AGENCY YAVAPAI CO. SO.  
DATE 03/30/15 TIME 2157  
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO BLACK CANYON

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS RAN 2 ALCOHOL FREE TESTS, THEN DID MY  
DEFICIENT & INVALID SAMPLE

SIGNATURE Kelly Wohlwend

INTOXILYZER 8000  
Location: YCSO - BLACK CANYON  
Serial Number: 80-000289  
Core Version: 8105.48  
03/30/2015 22:17:31

Standard Lot#: 25414100A4  
Last Changed By: L. MOODY #7486

QAS: K. WOHLWEND #9811  
YAUAPAI CO SO

Operator: K. WOHLWEND #9811  
YAUAPAI CO SO

Subject: TEST,SQAP  
DOB: 03/30/2000  
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:18:30
Diagnostic Test	Pass	22:19:08
Air Blank	0.000	22:19:35
0.100 Cal Check	0.100	22:19:55
Air Blank	0.000	22:20:25
Subject Test	XXX*	22:20:46
Air Blank	RFI**	22:20:57

\*Invalid Sample  
Begin new deprivation period  
\*\*RFI Detect

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YCSO - BLACK CANYON  
Serial Number: 80-000289  
Core Version: 8105.48  
03/30/2015 22:03:39

Standard Lot#: 25414100A4  
Last Changed By: L. MOODY #7486

QAS: K. WOHLWEND #9811  
YAUAPAI CO SO

Operator: K. WOHLWEND #9811  
YAUAPAI CO SO

Subject: TEST,SQAP  
DOB: 03/30/2000  
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:04:36
Diagnostic Test	Pass	22:05:13
Air Blank	0.000	22:05:40
0.100 Cal Check	0.101	22:06:01
Air Blank	0.000	22:06:31
Subject Test	0.007	22:07:42
Air Blank	0.000	22:08:10
Wait		
Air Blank	0.000	22:13:10
Subject Test	0.049*	22:16:17
Air Blank	0.000	22:16:50

\*Deficient Sample

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YCSO - BLACK CANYON  
Serial Number: 80-000289  
Core Version: 8105.48  
03/30/2015 21:57:36

Standard Lot#: 25414100A4  
Last Changed By: L. MOODY #7486

QAS: K. WOHLWEND #9811  
YAUAPAI CO SO

Operator: K. WOHLWEND #9811  
YAUAPAI CO SO

Subject: TEST,SQAP  
DOB: 03/30/2000  
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:59:49
Diagnostic Test	Pass	22:00:26
Air Blank	0.000	22:00:54
0.100 Cal Check	0.101	22:01:15
Air Blank	0.000	22:01:44
Subject Test	0.000	22:02:46
Air Blank	ABT*	22:02:54
Air Blank	1PS**	22:03:03

\*Sequence Aborted  
\*\*Improper Sample

Not a Successfully  
Completed  
Test Sequence

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST KELLY WOHLWEND AGENCY YAVAPAI CO SO  
DATE 03/30/15 TIME 2223  
INTOXILYZER SERIAL # 80-00289 LOCATION YCSO BLACK CANYON

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .1000 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 100 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Kelly Wohlwend 9811

INTOXILYZER 8000  
Location: YCSO - BLACK CANYON  
Serial Number: 80-000289  
Core Version: 8105.48  
03/30/2015 22:23:02

-----  
Standard Lot#: 25414100A4  
Last Changed By: L. MOODY #7486  
-----

DAS: K. WOHLWEND #9811  
YAVAPAI CO SO  
-----

Operator: K. WOHLWEND #9811  
YAVAPAI CO SO  
-----

Test	g/210L	Time
Air Blank	0.000	22:23:33
0.100 Cal Check	0.100	22:23:53
Air Blank	0.000	22:24:23

Preventative Maintenance  
Performed -- Timer Reset