

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST KELLY WOHLWEND AGENCY YAVAPAI COUNTY S.O.
DATE 05/28/15 TIME 9:38
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO BLACK CANYON

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .10 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 106 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Kelly Wohlwend 9811

INTOXILYZER 8000
Location: YCSO - BLACK CANYON
Serial Number: 80-000289
Core Version: 8105.48
05/28/2015 21:38:59

Standard Lot#: 25414100A4
Last Changed By: L. MOODY #7486

DAS: K. WOHLWEND #9811
YAVAPAI CO SO

Operator: K. WOHLWEND #9811
YAVAPAI CO SO

Test	g/210L	Time
Air Blank	0.000	21:39:58
0.100 Cal Check	0.106	21:40:19
Air Blank	0.000	21:40:49

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST KELLY WOHLWEED AGENCY YAVAPAI COUNTY S.O.
DATE 05/28/15 TIME 2143
INTOXILYZER SERIAL # 80-00289 LOCATION YESO BLACK CANYON

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. OC AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 10 AC. Result 0. 106 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Kelly Wohlwee 981

INTOXILYZER 8000
 Location: YCSO - BLACK CANYON
 Serial Number: 80-000289
 Core Version: 8105.48
 05/28/2015 21:43:17

Standard Lot#: 25414100A4
 Last Changed By: L. MOODY #7486

QAS: K. WOHLWEND #9811
 YAUAPAI CO SO

Operator: K. WOHLWEND #9811
 YAUAPAI CO SO

Subject: TEST,SOAP
 DOB: 05/28/2000
 Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:45:21
Diagnostic Test	Pass	21:45:58
Air Blank	0.000	21:46:26
0.100 Cal Check	0.100	21:46:46
Air Blank	0.000	21:47:16
Subject Test	0.000	21:47:44
Air Blank	ABT*	21:47:53
Air Blank	0.000	21:48:21

*Sequence Aborted

Not a Successfully
 Completed
 Test Sequence

INTOXILYZER 8000
 Location: YCSO - BLACK CANYON
 Serial Number: 80-000289
 Core Version: 8105.48
 05/26/2015 21:51:02

Standard Lot#: 25414100A4
 Last Changed By: L. MOODY #7486

QAS: K. WOHLWEND #9811
 YAUAPAI CO SO

Operator: K. WOHLWEND #9811
 YAUAPAI CO SO

Subject: TEST,SOAP
 DOB: 05/28/2000
 Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:52:51
Diagnostic Test	Pass	21:53:28
Air Blank	0.000	21:53:55
0.100 Cal Check	0.100	21:54:16
Air Blank	0.000	21:54:46
Subject Test	0.041*	21:57:53
Air Blank	0.000	21:58:25
Air Blank	0.000	21:58:52
Subject Test	XXX**	21:59:15
Air Blank	RFI***	21:59:32

*Deficient Sample
 **Invalid Sample
 Begin new deprivation period
 ***RFI Detect

Not a Successfully
 Completed
 Test Sequence

INTOXILYZER 8000
 Location: YCSO - BLACK CANYON
 Serial Number: 80-000289
 Core Version: 8105.48
 05/28/2015 21:59:55

Standard Lot#: 25414100A4
 Last Changed By: L. MOODY #7486

QAS: K. WOHLWEND #9811
 YAUAPAI CO SO

Operator: K. WOHLWEND #9811
 YAUAPAI CO SO

Subject: TEST,SOAP
 DOB: 05/28/2000
 Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	22:00:46
Air Blank	0.000	22:01:15

*Improper Sample

Not a Successfully
 Completed
 Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST KELLY WOHLVEND AGENCY YAVAPAI COUNTY SO
DATE 05/28/15 TIME 2202
INTOXILYZER SERIAL # 80-000 289 LOCATION YCSO BLACK CANYON

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .10 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 100 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Kelly Wohlwend 9811

INTOXILYZER 8000

Location: YCSO - BLACK CANYON

Serial Number: 60-000289

Core Version: 8105.48

05/26/2015

22:02:07

Standard Lot#: 25414100A4

Last Changed By: L. MOODY #7486

DAS: K. WOHLWEND #9811

YAVAPAI CO SO

Operator: K. WOHLWEND #9811

YAVAPAI CO SO

Test	g/210L	Time
Air Blank	0.000	22:02:38
0.100 Cal Check	0.100	22:02:58
Air Blank	0.000	22:03:28

Preventative Maintenance
Performed -- Timer Reset