

17-033320

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Logan Moody #7486 AGENCY Yavapai County Sheriff's Office
DATE 09/16/2017 TIME 1826
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO - Black Canyon

- (E) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (E) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (E) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (E) 4. Air blank completed.
- (E) 5. Calibration check completed. Test results 0. 100 AC.
- (E) 6. Air blank completed.
- (E) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Logan Moody #7486

17-033320

INTOXILYZER 8000
Location: YCSO - BLACK CANYON
Serial Number: 80-000289
Core Version: 8105.48
09/16/2017 18:26:12

Standard Lot#: 23016100A1
Last Changed By: L. MOODY #7486

QAS: L. MOODY #7486
YAVAPAI CSO

Operator: L. MOODY #7486
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	18:26:41
0.100 Cal Check	0.100	18:27:01
Air Blank	0.000	18:27:31

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Logan Moody #7486 AGENCY Yavapai County Sheriff's Office
DATE 9/16/2017 TIME 1912
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO - Black Canyon

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 100 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Logan Moody #7486

INTOXILYZER 8000
Location: YCSO - BLACK CANYON
Serial Number: 80-000289
Core Version: 8105.48
09/16/2017 18:12:53

Standard Lot#: 23016100A1
Last Changed By: L. MOODY #7486

QAS: L. MOODY #7486
YAVAPAI CSO

Operator: L. MOODY #7486
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	18:13:22
0.100 Cal Check	0.100	18:13:42
Air Blank	0.000	18:14:12

Preventative Maintenance
Performed -- Timer Reset

INTOXILYZER 8000
Location: YCSO - BLACK CANYON
Serial Number: 80-000289
Core Version: 8105.48
09/16/2017 18:12:53

Standard Lot#: 23016100A1
Last Changed By: L. MOODY #7486

QAS: L. MOODY #7486
YAVAPAI CSO

Operator: L. MOODY #7486
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	18:13:22
0.100 Cal Check	0.100	18:13:42
Air Blank	0.000	18:14:12

Preventative Maintenance
Performed -- Timer Reset

17-033320

INTOXILYZER 8000

Location: YCSO - BLACK CANYON

Serial Number: 80-000289

Core Version: 8105.48

09/16/2017

18:14:56

Standard Lot#: 23016100A1

Last Changed By: L. MOODY

#7486

QAS: L. MOODY

#7486

YAVAPAI CSO

Operator: L. MOODY

#7486

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1990

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:15:44
Diagnostic Test	Pass	18:16:19
Air Blank	0.000	18:16:47
0.100 Cal Check	0.099	18:17:07
Air Blank	0.000	18:17:37
Subject Test	0.000	18:17:57
Air Blank	ABT*	18:18:05
Air Blank	RFI**	18:18:12

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

17-033320

INTOXILYZER 8000
Location: YCSO - BLACK CANYON
Serial Number: 80-000289
Core Version: 8105.48
09/16/2017 18:18:22

Standard Lot#: 23016100A1
Last Changed By: L. MOODY #7486

QAS: L. MOODY #7486
YAVAPAI CSO

Operator: L. MOODY #7486
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1990
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:19:13
Diagnostic Test	Pass	18:19:48
Air Blank	0.000	18:20:16
0.100 Cal Check	0.100	18:20:37
Air Blank	0.000	18:21:06
Subject Test	0.070*	18:24:14
Air Blank	0.000	18:24:45
Air Blank	0.000	18:25:12
Subject Test	XXX**	18:25:45
Air Blank	IPS***	18:25:52

*Deficient Sample
**Invalid Sample
 Begin new deprivation period
***Improper Sample

Not a Successfully Completed Test Sequence