

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Cpl. MICHAEL D. MORRIS AGENCY PVPP  
DATE 11-06-09 TIME 1545  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPP BOOKING ROOM

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution N/A AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.098 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Cpl. M. Morris # 50 QAS# 33034

DPS Form Exh G-2 (Rev 05-01)

RECEIVED Intox 8000 BACK FROM DPS ALCOHOL LAB FROM REPAIRS.  
NO OTHER QUALITY ASSURANCE SPECIALIST ON DUTY.

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

11/06/2009

15:45:28

-----  
Standard Lot#: 804402I

Last Changed By: M. MORRIS #50  
-----

QAS: M. MORRIS #50

PRESCOTT VALLEY PD  
-----

| Test      | g/210L       | Time     |
|-----------|--------------|----------|
| Air Blank | 0.000        | 15:45:56 |
| Cal Check | <u>0.098</u> | 15:46:14 |
| Air Blank | 0.000        | 15:46:43 |

QAS# 33034

Cpl. M. P. D. M. #50  
Operator's Signature

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Cpl. MICHAEL D. MORRIS AGENCY PVPD  
DATE 11-06-09 TIME 1548  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPD BOOKING ROOM

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000% AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 010% 100 AC. Result 0. 096% AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

SIGNATURE Cpl. M. PD. M. #50 QAS# 33034

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number : 80-000293  
11/06/2009

15:48:47

---

Standard Lot#: 804402I  
Last Changed By: M. MORRIS #50

---

QAS: M. MORRIS #50  
PRESCOTT VALLEY PD

---

Operator: M. MORRIS #50  
PRESCOTT VALLEY PD

---

Subject: SQAP,TEST  
DOB: 04/07/1974  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

| Test             | g/210L | Time     |
|------------------|--------|----------|
| Air Blank        | 0.000  | 15:50:05 |
| Diagnostic Test  | Pass   | 15:50:32 |
| Air Blank        | 0.000  | 15:51:00 |
| 0.100 Cal Check  | 0.097  | 15:51:19 |
| Air Blank        | 0.000  | 15:51:48 |
| Subject Test     | 0.000  | 15:52:05 |
| Air Blank        | 0.000  | 15:52:32 |
| Five Minute Wait | ABT*   |          |
| Air Blank        | 0.000  | 15:53:07 |

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number : 80-000293  
11/06/2009

15:53:34

---

Standard Lot#: 804402I  
Last Changed By: M. MORRIS #50

---

QAS: M. MORRIS #50  
PRESCOTT VALLEY PD

---

Operator: M. MORRIS #50  
PRESCOTT VALLEY PD

---

Subject: SQAP,TEST  
DOB: 04/07/1974  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | IPS*   | 15:54:20 |
| Air Blank | 0.000  | 15:54:49 |

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number : 80-000293

11/06/2009

15:54:58

---

Standard Lot#: 804402I

Last Changed By: M. MORRIS #50

---

QAS: M. MORRIS #50

PRESCOTT VALLEY PD

---

Operator: M. MORRIS #50

PRESCOTT VALLEY PD

---

Subject: SQAP,TEST

DOB: 04/07/1974

Sex: M

Weight: 200

---

15 Minute Deprivation Period? Yes

---

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | RFI*   | 15:55:45 |
| Air Blank | 0.000  | 15:56:13 |

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number : 80-000293  
11/06/2009 15:56:18

---

Standard Lot#: 804402I  
Last Changed By: M. MORRIS #50

---

QAS: M. MORRIS #50  
PRESCOTT VALLEY PD

---

Operator: M. MORRIS #50  
PRESCOTT VALLEY PD

---

Subject: SQAP, TEST  
DOB: 04/07/1974  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 15:57:12 |
| Diagnostic Test | Pass   | 15:57:39 |
| Air Blank       | 0.000  | 15:58:07 |
| 0.100 Cal Check | 0.101  | 15:58:25 |
| Air Blank       | 0.000  | 15:58:55 |
| Subject Test    | 0.083* | 16:02:03 |
| Air Blank       | 0.000  | 16:02:35 |
| Air Blank       | 0.000  | 16:03:02 |
| Subject Test    | XXX**  | 16:03:29 |
| Air Blank       | 0.000  | 16:04:01 |

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Cpl. MICHAEL D. MORRIS AGENCY PVPO  
DATE 11-06-09 TIME 1605  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPO BOOKING ROOM

- AMS 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution N/A AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- AMS 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- AMS 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- AMS 4. Air blank completed.
- AMS 5. Calibration check completed. Test results 0. 096 AC.
- AMS 6. Air blank completed.
- AMS 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Cpl. M. D. Morris #50 QAS# 33034

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

11/06/2009

16:05:08

-----  
Standard Lot#: 804402I

Last Changed By: M. MORRIS #50

-----  
QAS: M. MORRIS #50

PRESCOTT VALLEY PD  
-----

| Test      | g/210L       | Time     |
|-----------|--------------|----------|
| Air Blank | 0.000        | 16:05:35 |
| Cal Check | <u>0.096</u> | 16:05:54 |
| Air Blank | 0.000        | 16:06:23 |

Cpl. M. MORRIS #50 QAS# 33034  
Operator's Signature