

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Krogedal AGENCY PVPD  
DATE 8/11/11 TIME 0402  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPD

(~~Ch~~ 1. Display Reads "PUSH BUTTON TO START")

DIAGNOSTIC TESTS

(~~Ch~~ 1. Clock time check.  
(~~Ch~~ 2. Date check.

OPERATIONAL TESTS

(~~Ch~~ 1. Alcohol-free subject test result 0. 000 AC.  
(~~Ch~~ 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(~~Ch~~ 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(~~Ch~~ 4. Standard Calibration Check standard 0. 094 AC. Result 0. 094 AC.

Instrument is operating properly and accurately. ' YES  NO

COMMENTS

SIGNATURE Ch Krogedal #90 / QAS # 33200

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

08/11/2011

16:00:24

---

Standard Lot#: 779386

Last Changed By: C. KROGEDAL #90

---

QAS: C. KROGEDAL #90

PRESCOTT VALLEY PD

---

Operator: C. KROGEDAL #90

PRESCOTT VALLEY PD

---

Subject: SQAP,TEST

DOB: 01/15/1967

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	16:01:27
Diagnostic Test	Pass	16:02:01
Air Blank	0.000	16:02:28
0.100 Cal Check	0.094	16:02:49
Air Blank	0.000	16:03:18
Subject Test	0.000	16:03:40
Air Blank	0.000	16:04:08
Wait	ABT*	
Air Blank	0.000	16:04:45

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number: 80-000293  
Core Version: 8105.48  
08/11/2011 16:04:56

---

Standard Lot#: 779386  
Last Changed By: C. KROGEDAL #90

---

QAS: C. KROGEDAL #90  
PRESCOTT VALLEY PD

---

Operator: C. KROGEDAL #90  
PRESCOTT VALLEY PD

---

Subject: SQAP,TEST  
DOB: 01/15/1956  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	16:05:37
Air Blank	0.000	16:06:05

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

08/11/2011

16:06:34

---

Standard Lot#: 779386

Last Changed By: C. KROGEDAL #90

---

QAS: C. KROGEDAL #90

PRESCOTT VALLEY PD

---

Operator: C. KROGEDAL #90

PRESCOTT VALLEY PD

---

Subject: SQAP,TEST

DOB: 01/15/1957

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	16:07:29
Air Blank	0.000	16:07:58

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number: 80-000293  
Core Version: 8105.48  
08/11/2011 16:08:22

---

Standard Lot#: 779386  
Last Changed By: C. KROGEDAL #90

---

QAS: C. KROGEDAL #90  
PRESCOTT VALLEY PD

---

Operator: C. KROGEDAL #90  
PRESCOTT VALLEY PD

---

Subject: SQAP,TEST  
DOB: 01/15/1956  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	16:09:31
Diagnostic Test	Pass	16:10:05
Air Blank	0.000	16:10:32
0.100 Cal Check	0.094	16:10:53
Air Blank	0.000	16:11:23
Subject Test	0.244*	16:14:30
Air Blank	0.000	16:15:02
Air Blank	0.000	16:15:29
Subject Test	XXX**	16:15:47
Air Blank	0.000	16:16:20

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Krogedel AGENCY PVPD  
DATE 8/11/11 TIME 1555  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPD

() 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

() 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

() 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

() 4. Air blank completed.

() 5. Calibration check completed. Test results 0.095 AC.

() 6. Air blank completed.

() 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

C. Krogedel #90 / GAS # 33200

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

08/11/2011

15:55:51

-----  
Standard Lot#: 779386

Last Changed By: C. KROGEDAL #90

-----  
QAS: C. KROGEDAL #90

PRESCOTT VALLEY PD

-----  
Operator: C. KROGEDAL #90

PRESCOTT VALLEY PD

-----  
Test                      g/210L                      Time  
-----  
Air Blank                      0.000                      15:56:22  
0.100 Cal Check                0.095                      15:56:43  
Air Blank                      0.000                      15:57:13

Preventative Maintenance  
Performed -- Timer Reset