

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY PRESOTT VALLEY PD  
DATE 6-19-15 TIME 2133  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPD/BOOKING ROOM

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE \_\_\_\_\_  

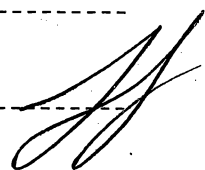

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number: 80-000293  
Core Version: 8105.48  
06/19/2015

21:33:49

-----  
Standard Lot#: 26614100A6  
Last Changed By: A. LAWRENCE #61  
-----

QAS: A. LAWRENCE #61  
PRESCOTT VALLEY PD  
-----

Operator: A. LAWRENCE #61  
PRESCOTT VALLEY PD  
-----



Test	g/210L	Time
Air Blank	0.000	21:34:19
0.100 Cal Check	0.099	21:34:39
Air Blank	0.000	21:35:09

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY PRESOTT VALLEY PD  
DATE 6-19-15 TIME 2135  
INTOXILYZER SERIAL # 80-000293 LOCATION PUPD / Booking Room

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.  
(  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.  
(  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(  ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

INTOX 8000 WORKING WELL. DPS PREVENTATIVE MAINTENANCE IS THE ONLY REASON INTOX IS BEING TAKEN OUT OF SERVICE AND SENT TO PHOENIX, DPS.

SIGNATURE [Signature] #61

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number: 80-000293  
Core Version: 8105.48  
06/19/2015

21:35:42

---

Standard Lot#: 26614100A6  
Last Changed By: A. LAWRENCE #61

---

QAS: A. LAWRENCE #61  
PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE #61  
PRESCOTT VALLEY PD

---

Subject: SQAP, TEST  
DOB: 01/05/1980  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	21:36:49
Diagnostic Test	Pass	21:37:23
Air Blank	0.000	21:37:50
0.100 Cal Check	0.099	21:38:11
Air Blank	0.000	21:38:40
Subject Test	0.000	21:39:45
Air Blank	0.000	21:40:13
Wait	ABT*	
Air Blank	0.000	21:40:47

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number: 80-000293  
Core Version: 8105.48  
06/19/2015 21:40:53

---

Standard Lot#: 26614100A6  
Last Changed By: A. LAWRENCE #61

---

QAS: A. LAWRENCE #61  
PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE #61  
PRESCOTT VALLEY PD

---

Subject: SQAP, TEST  
DOB: 01/05/1980  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	21:41:30
Air Blank	0.000	21:41:59

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

06/19/2015

21:42:03

---

Standard Lot#: 26614100A6

Last Changed By: A. LAWRENCE #61

---

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Subject: SQAP, TEST

DOB: 01/05/1980

Sex: M

Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	21:42:39
Air Blank	0.000	21:43:08

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number: 80-000293  
Core Version: 8105.48  
06/19/2015

21:43:11

---

Standard Lot#: 26614100A6  
Last Changed By: A. LAWRENCE #61

---

QAS: A. LAWRENCE #61  
PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE #61  
PRESCOTT VALLEY PD



---

Subject: SQAP, TEST  
DOB: 01/05/1980  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	21:43:59
Diagnostic Test	Pass	21:44:33
Air Blank	0.000	21:45:00
0.100 Cal Check	0.098	21:45:21
Air Blank	0.000	21:45:50
Subject Test	0.205*	21:48:58
Air Blank	0.000	21:49:30
Air Blank	0.000	21:49:57
Subject Test	XXX**	21:50:24
Air Blank	0.000	21:50:58

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

06/19/2015

21:51:16

-----  
Standard Lot#: 26614100A6

Last Changed By: A. LAWRENCE #61

-----  
QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

-----  
Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD  
-----

Test	g/210L	Time
Air Blank	0.000	21:51:45
0.100 Cal Check	0.098	21:52:06
Air Blank	0.000	21:52:36

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY PRESOTT VALLEY PD  
DATE 6-19-15 TIME 2:51  
INTOXILYZER SERIAL # 80-000293 LOCATION PUPD/BOOKING

- ( 4 ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- ( 5 ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( 5 ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( 4 ) 4. Air blank completed.
- ( 4 ) 5. Calibration check completed. Test results 0. 098 AC.
- ( 4 ) 6. Air blank completed.
- ( 4 ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE \_\_\_\_\_  
*[Signature]* #61