

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY PRESOTT VALLEY PD  
DATE 7-28-15 TIME 1951 HOURS  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPD/BOOKING

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 096 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

\* SEE ATTACHED DPS INTOXILYZER  
MAINTENANCE REPORT. ~~BT~~

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

07/28/2015

19:51:37

-----  
Standard Lot#: 26614100A6

Last Changed By: A. LAWRENCE #61

-----  
QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

-----  
Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD  
-----

Test	g/210L	Time
Air Blank	0.000	19:52:08
0.100 Cal Check	0.096	19:52:29
Air Blank	0.000	19:52:58

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY PRESOTT VALLEY PD  
DATE 7-28-15 TIME 1953 Hours  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPD/BOOKING

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.  
(  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.  
(  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(  ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES \_\_\_\_\_ NO \_\_\_\_\_

COMMENTS

INTOX PUT BACK INTO SERVICE (RECEIVED BACK  
FROM DPS / YEARLY PREVENTATIVE MAINTENANCE)  
NO ISSUES / BACK IN SERVICE.

SIGNATURE \_\_\_\_\_

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

07/28/2015

19:53:18

---

Standard Lot#: 26614100A6

Last Changed By: A. LAWRENCE #61

---

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Subject: SQAP, TEST

DOB: 01/05/1980

Sex: M

Weight: 200

---

15 Minute Deprivation Period? ~~Yes~~ 

---

Test	g/210L	Time
Air Blank	0.000	19:54:14
Diagnostic Test	Pass	19:54:48
Air Blank	0.000	19:55:15
0.100 Cal Check	0.098	19:55:36
Air Blank	0.000	19:56:06
Subject Test	0.000	19:56:26
Air Blank	0.000	19:56:53
Wait	ABT*	
Air Blank	0.000	19:57:28

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

07/28/2015

19:57:33

---

Standard Lot#: 26614100A6

Last Changed By: A. LAWRENCE #61

---

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD


---

Subject: SQAP, TEST

DOB: 01/05/1980

Sex: M

Weight: 200

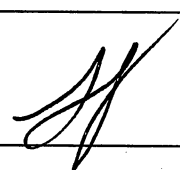

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	19:58:07
Air Blank	0.000	19:58:36

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

07/28/2015

19:58:40

---

Standard Lot#: 26614100A6

Last Changed By: A. LAWRENCE #61

---

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Subject: SQAP, TEST

DOB: 01/05/1980

Sex: M

Weight: 200



---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	19:59:14
Air Blank	0.000	19:59:42

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

07/28/2015

19:59:46

---

Standard Lot#: 26614100A6

Last Changed By: A. LAWRENCE #61

---

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Subject: SQAP, TEST

DOB: 01/05/1980

Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	20:00:40
Diagnostic Test	Pass	20:01:14
Air Blank	0.000	20:01:42
0.100 Cal Check	0.097	20:02:03
Air Blank	0.000	20:02:32
Subject Test	0.090*	20:05:40
Air Blank	0.000	20:06:12
Air Blank	0.000	20:06:40
Subject Test	XXX**	20:06:55
Air Blank	0.000	20:07:29

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period



**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY Prescott Valley PD  
DATE 7-28-15 TIME 2007 Hours  
INTOXILYZER SERIAL # 80-000293 LOCATION FVPD/Booking Room

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 095 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

07/28/2015

20:07:54

-----  
Standard Lot#: 26614100A6

Last Changed By: A. LAWRENCE #61

-----  
QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

-----  
Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD  
-----

Test	g/210L	Time
Air Blank	0.000	20:08:23
0.100 Cal Check	0.095	20:08:44
Air Blank	0.000	20:09:13

Preventative Maintenance  
Performed -- Timer Reset



ARIZONA DEPARTMENT OF PUBLIC SAFETY  

---

INTOXILYZER MAINTENANCE REPORT

CR NO. I800002930002

Page 1 of 1

AGENCY Prescott Valley Police Department  
Prescott Valley, AZ 86314  
OFFICER LAWRENCE, #61  
DATE July 24, 2015



**INSTRUMENT**

80000293

**INSTRUMENT LOCATION**

Prescott Valley Police Department

**MAINTENANCE REQUESTED**

Repair and Preventative Maintenance

**MAINTENANCE PERFORMED**

***Parts Replaced:***

Breath Hose Check Valve, End Block Check Valve, Source  
Source Block

***Preventative Maintenance Performed:***

Inspection, Calibration, Verification, Adjustment

A handwritten signature in cursive script, appearing to read "Joseph M Slowinski".

---

JOSEPH M SLOWINSKI, #6101, Electronics Technician  
Central Regional Crime Laboratory  
2102 W. Encanto Blvd., Phoenix, AZ 85009  
602-223-2159