

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY PRESOTT VALLEY PD  
DATE 5/18/2016 TIME 2:22  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPD/Booking Room

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE \_\_\_\_\_  
*T. Lawrence*

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

05/18/2016

21:22:59

-----  
Standard Lot#: 26614100A6


Last Changed By: A. LAWRENCE #61

-----  
QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

-----  
Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD  
-----



Test	g/210L	Time
Air Blank	0.000	21:23:29
0.100 Cal Check	0.099	21:23:50
Air Blank	0.000	21:24:19

Preventative Maintenance  
Performed -- Timer Reset