

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY Prescott Valley PD  
DATE 7/16/16 TIME 0003  
INTOXILYZER SERIAL # 80-000293 LOCATION PV PD/Booking

- ( 4 ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- ( 4 ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( 4 ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( 4 ) 4. Air blank completed.
- ( 4 ) 5. Calibration check completed. Test results 0. 097 AC.
- ( 4 ) 6. Air blank completed.
- ( 4 ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE \_\_\_\_\_

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

07/17/2016

00:03:05

-----  
Standard Lot#: 06616100A1

Last Changed By: A. LAWRENCE #61  
-----

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD  
-----

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD  
-----

Test	g/210L	Time
Air Blank	0.000	00:03:36
0.100 Cal Check	0.097	00:03:57
Air Blank	0.000	00:04:26

Preventative Maintenance  
Performed -- Timer Reset



EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST T. LAURENCE AGENCY Prescott Valley PD  
DATE 7/16/16 TIME 0005  
INTOXILYZER SERIAL # 80-000 LOCATION PV PD Booking Room

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.  
(  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.  
(  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(  ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

Intox Working Well

SIGNATURE  #11

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

07/17/2016

00:05:55

---

Standard Lot#: 06616100A1

Last Changed By: A. LAWRENCE #61

---

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Subject: SQAP, TEST

DOB: 01/05/1980

Sex: M

Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	00:07:10
Diagnostic Test	Pass	00:07:44
Air Blank	0.000	00:08:11
0.100 Cal Check	0.097	00:08:32
Air Blank	0.000	00:09:02
Subject Test	0.000	00:09:26
Air Blank	0.000	00:09:53
Wait	ABT*	
Air Blank	0.000	00:10:31

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

07/17/2016

00:10:38

---

Standard Lot#: 06616100A1

Last Changed By: A. LAWRENCE #61

---

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Subject: SQAP, TEST

DOB: 01/05/1980

Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	00:11:22
Air Blank	0.000	00:11:50

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

07/17/2016

00:11:55

---

Standard Lot#: 06616100A1

Last Changed By: A. LAWRENCE #61

---

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD

---

Subject: SQAP,TEST

DOB: 01/05/1980

Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	00:12:29
Air Blank	0.000	00:12:57

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number: 80-000293  
Core Version: 8105.48  
07/17/2016

00:13:01

---

Standard Lot#: 06616100A1  
Last Changed By: A. LAWRENCE #61

---

QAS: A. LAWRENCE #61  
PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE #61  
PRESCOTT VALLEY PD

---

Subject: SQAP, TEST  
DOB: 01/05/1980  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	00:13:51
Diagnostic Test	Pass	00:14:25
Air Blank	0.000	00:14:52
0.100 Cal Check	0.097	00:15:13
Air Blank	0.000	00:15:43
Subject Test	0.292*	00:18:50
Air Blank	0.000	00:19:23
Air Blank	0.000	00:19:51
Subject Test	RGE**	00:20:20
Air Blank	0.000	00:20:55

\*Deficient Sample

\*\*Range Exceeded

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY Prescott Valley PD  
DATE 7/16/16 TIME 0021  
INTOXILYZER SERIAL # 80-000293 LOCATION PVDD / Booking Room

- ( 5 ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- ( 5 ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( 5 ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( 4 ) 4. Air blank completed.
- ( 4 ) 5. Calibration check completed. Test results 0. 0916 AC.
- ( 5 ) 6. Air blank completed.
- ( 4 ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE \_\_\_\_\_  


INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

07/17/2016

00:21:25

-----  
Standard Lot#: 06616100A1

Last Changed By: A. LAWRENCE #61  
-----

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD  
-----

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD  
-----

Test	g/210L	Time
Air Blank	0.000	00:21:56
0.100 Cal Check	0.096	00:22:17
Air Blank	0.000	00:22:47

Preventative Maintenance  
Performed -- Timer Reset

