

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY PRESCOTT VALLEY PD
DATE 4-22-17 TIME 2306
INTOXILYZER SERIAL # 80-000293 LOCATION PRESCOTT VALLEY PD / BOOKING ROOM.

- (4) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (4) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (4) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (4) 4. Air blank completed.
- (4) 5. Calibration check completed. Test results 0. 098 AC.
- (4) 6. Air blank completed.
- (4) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____ #61

DPS Form Exh G-2 (Rev 05-01)

INTOX WAS RETURNED FROM DPS. NO CALIBRATION ISSUES ACCORDING THEM. PREVENTATIVE MAINTENANCE ONLY. #61

Lawrence

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

04/22/2017

23:06:02

Standard Lot#: 06616100A1

Last Changed By: A. LAWRENCE #61

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD

Test	g/210L	Time
Air Blank	0.000	23:06:31
0.100 Cal Check	0.098	23:06:52
Air Blank	0.000	23:07:21

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. LAWRENCE AGENCY PRESOTT VALLEY

DATE 4-22-17 TIME 2307

INTOXILYZER SERIAL # 80-000293 LOCATION PUPD/BOOKING ROOM

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

NO ISSUES OPERATING CORRECTLY

SIGNATURE [Signature] #01

INTOXILYZER 8000
Location: PRESCOTT VALLEY PD
Serial Number: 80-000293
Core Version: 8105.48
04/22/2017 23:07:46

Standard Lot#: 06616100A1
Last Changed By: A. LAWRENCE #61

QAS: A. LAWRENCE #61
PRESCOTT VALLEY PD

Operator: A. LAWRENCE #61
PRESCOTT VALLEY PD

Subject: SQAP, TEST
DOB: 01/05/1980
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:08:38
Diagnostic Test	Pass	23:09:12
Air Blank	0.000	23:09:40
0.100 Cal Check	0.099	23:10:01
Air Blank	0.000	23:10:30
Subject Test	0.000	23:10:51
Air Blank	0.000	23:11:18
Wait	ABT*	
Air Blank	0.000	23:11:53

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT VALLEY PD
Serial Number: 80-000293
Core Version: 8105.48
04/22/2017 23:11:58

Standard Lot#: 06616100A1
Last Changed By: A. LAWRENCE #61

QAS: A. LAWRENCE #61
PRESCOTT VALLEY PD

Operator: A. LAWRENCE #61
PRESCOTT VALLEY PD



Subject: SQAP, TEST
DOB: 01/05/1980
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	23:12:34
Air Blank	0.000	23:13:02

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

04/22/2017

23:13:07

Standard Lot#: 06616100A1

Last Changed By: A. LAWRENCE #61

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD

Subject: SQAP, TEST

DOB: 01/05/1980

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	23:13:46
Air Blank	0.000	23:14:15

*RFI Detect



Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

04/22/2017

23:14:18

Standard Lot#: 06616100A1

Last Changed By: A. LAWRENCE #61

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD

Subject: SQAP, TEST

DOB: 01/05/1980

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:15:06
Diagnostic Test	Pass	23:15:40
Air Blank	0.000	23:16:07
0.100 Cal Check	0.099	23:16:28
Air Blank	0.000	23:16:58
Subject Test	0.389*	23:20:05
Air Blank	0.000	23:20:37
Air Blank	0.000	23:21:05
Subject Test	XXX**	23:21:33
Air Blank	0.000	23:22:07

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY PRESCOTT VALLEY
DATE 4-22-17 TIME 2322
INTOXILYZER SERIAL # 80-000293 LOCATION PVPD/BOOKING ROOM

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 097 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____ [Signature] #61

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

04/22/2017

23:22:21

Standard Lot#: 06616100A1

Last Changed By: A. LAWRENCE #61

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD



Test	g/210L	Time
Air Blank	0.000	23:22:50
0.100 Cal Check	0.097	23:23:11
Air Blank	0.000	23:23:41

Preventative Maintenance
Performed -- Timer Reset



ARIZONA DEPARTMENT OF PUBLIC SAFETY

INTOXILYZER MAINTENANCE REPORT

CR NO. 1800002930003

Page 1 of 1

AGENCY Prescott Valley Police Department
Prescott Valley, AZ 86314
OFFICER LAWRENCE, #61
DATE April 17, 2017



INSTRUMENT

80000293

INSTRUMENT LOCATION

Prescott Valley Police Department

MAINTENANCE REQUESTED

Preventative Maintenance

MAINTENANCE PERFORMED

Preventative Maintenance Performed:

Inspection, Calibration, Verification

Unable to duplicate low calibration check.

A handwritten signature in black ink, appearing to read "Robert W. Stephenson".

ROBERT W. STEPHENSON, #5742, Forensic Scientist
Central Regional Crime Laboratory
2102 W. Encanto Blvd., Phoenix, AZ 85009
602-223-2114