

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST B PENNER #29 AGENCY PRESOTT VALLEY P.D.
DATE 6-22-17 TIME 1004
INTOXILYZER SERIAL # 80-000293 LOCATION PVPO MAIN-HOLDING

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
~~OR~~
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  #29

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

06/22/2017

10:08:22

Standard Lot#: 06616100A1

Last Changed By: A. LAWRENCE #61

QAS: B. PENNER #29

PRESCOTT VALLEY PD

Operator: B. PENNER #29

PRESCOTT VALLEY PD

Test	g/210L	Time
Air Blank	0.000	10:08:59
0.100 Cal Check	0.099	10:09:20
Air Blank	0.000	10:09:49

Preventative Maintenance
Performed -- Timer Reset