

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST B. PENNER #29 AGENCY PRESCOTT VALLEY PD  
DATE 11-10-17 TIME 1821  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPO-MAIN HOLDING

(*cl*) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .10 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator.

Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34 \cdot C \pm 0.2 \cdot C$

- (*cl*) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(*cl*) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(*cl*) 4. Air blank completed.  
(*cl*) 5. Calibration check completed. Test results 0. .099 AC.  
(*cl*) 6. Air blank completed.  
(*cl*) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *B. Penner #29*

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

11/10/2017

18:22:48

-----  
Standard Lot#: 06117100A2

Last Changed By: B. PENNER #29

-----  
QAS: B. PENNER #29

PRESCOTT VALLEY PD

-----  
Operator: B. PENNER #29

PRESCOTT VALLEY PD

-----  
Test                                   g/210L                                   Time  
-----  
Air Blank                               0.000                                   18:23:22  
0.100 Cal Check                       0.099                                   18:23:43  
Air Blank                               0.000                                   18:24:12

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST B. PENNER #29 AGENCY PRESLOTT VALLEY PD  
DATE 11-10-17 TIME 1821  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPO MAIN - HOLDING

(M) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (M) 1. Clock time check.  
(M) 2. Date check.

OPERATIONAL TESTS

- (M) 1. Alcohol-free subject test result 0. 000 AC.  
(M) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(M) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(M) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES  NO

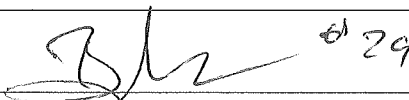
COMMENTS

---

---

---

---

SIGNATURE  #29

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number: 80-000293  
Core Version: 8105.48  
11/10/2017

18:33:23

---

Standard Lot#: 06117100A2  
Last Changed By: B. PENNER #29

---

QAS: B. PENNER #29  
PRESCOTT VALLEY PD

---

Operator: B. PENNER #29  
PRESCOTT VALLEY PD

---

Subject: SQAP,TEST  
DOB: 10/01/2017  
Sex: M Weight: 123

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	18:35:22
Diagnostic Test	Pass	18:35:56
Air Blank	0.000	18:36:23
0.100 Cal Check	0.099	18:36:44
Air Blank	0.000	18:37:13
Subject Test	0.000	18:37:34
Air Blank	0.000	18:38:02
Wait	ABT*	
Air Blank	0.000	18:39:06

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number: 80-000293  
Core Version: 8105.48  
11/10/2017 18:41:52

---

Standard Lot#: 06117100A2  
Last Changed By: B. PENNER #29

---

QAS: B. PENNER #29  
PRESCOTT VALLEY PD

---

Operator: B. PENNER #29  
PRESCOTT VALLEY PD

---

Subject: SQAP, TEST  
DOB: 01/01/2017  
Sex: M Weight: 123

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	18:42:45
Air Blank	0.000	18:43:14

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number: 80-000293  
Core Version: 8105.48  
11/10/2017 18:43:52

---

Standard Lot#: 06117100A2  
Last Changed By: B. PENNER #29

---

QAS: B. PENNER #29  
PRESCOTT VALLEY PD

---

Operator: B. PENNER #29  
PRESCOTT VALLEY PD

---

Subject: SQAP,TEST  
DOB: 01/01/2017  
Sex: M Weight: 123

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	18:44:43
Air Blank	0.000	18:45:11

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number: 80-000293  
Core Version: 8105.48  
11/10/2017 18:46:41

---

Standard Lot#: 06117100A2  
Last Changed By: B. PENNER #29

---

QAS: B. PENNER #29  
PRESCOTT VALLEY PD

---

Operator: B. PENNER #29  
PRESCOTT VALLEY PD

---

Subject: SQAP, TEST  
DOB: 01/01/2017  
Sex: M Weight: 123

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	18:47:41
Diagnostic Test	Pass	18:48:15
Air Blank	0.000	18:48:43
0.100 Cal Check	0.098	18:49:04
Air Blank	0.000	18:49:33
Subject Test	0.223*	18:52:41
Air Blank	0.000	18:53:13
Air Blank	0.000	18:53:40
Subject Test	XXX**	18:54:15
Air Blank	0.000	18:54:54

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST B. PENNER #29 AGENCY PRESOTT VALLEY PD  
DATE 11-10-17 TIME 1902  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPD MAIN - HOLDING

(M) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
~~OR~~

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator.

Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34 \pm 0.2 \text{ C}$

- (B) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(B) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(B) 4. Air blank completed.  
(B) 5. Calibration check completed. Test results 0. 098 AC.  
(B) 6. Air blank completed.  
(A) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] #29

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

11/10/2017

19:03:39

-----  
Standard Lot#: 06117100A2

Last Changed By: B. PENNER #29  
-----

QAS: B. PENNER #29

PRESCOTT VALLEY PD  
-----

Operator: B. PENNER #29

PRESCOTT VALLEY PD  
-----

Test	g/210L	Time
Air Blank	0.000	19:04:10
0.100 Cal Check	0.098	19:04:31
Air Blank	0.000	19:05:01

Preventative Maintenance  
Performed -- Timer Reset