

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY PRESOTT VALLEY PD  
DATE 2-1-18 TIME 2329  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPD/Booking Room

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 099 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

DPS Form Exh G-2 (Rev 05-01)

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

02/01/2018

23:29:45

-----  
Standard Lot#: 06117100A2

Last Changed By: B. PENNER #29

-----  
QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD

-----  
Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD

-----  
Test                      g/210L                      Time  
-----  
Air Blank                      0.000                      23:30:15  
0.100 Cal Check              0.099                      23:30:35  
Air Blank                      0.000                      23:31:05

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY PRESOTT VALLEY PD  
DATE 2-1-18 TIME 2331  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPD / BOOKING ROOM

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.  
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.  
() 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
() 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

INTOX WORKING WITHIN STANDARDS.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

02/01/2018

23:31:56

---

Standard Lot#: 06117100A2

Last Changed By: B. PENNER

#29

---

QAS: A. LAWRENCE

#61

PRESCOTT VALLEY PD

---

Operator: A. LAWRENCE

#61

PRESCOTT VALLEY PD

---

Subject: SQAP, TEST

DOB: 01/05/1980

Sex: M

Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	23:33:00
Diagnostic Test	Pass	23:33:34
Air Blank	0.000	23:34:02
0.100 Cal Check	0.099	23:34:22
Air Blank	0.000	23:34:52
Subject Test	0.000	23:35:18
Air Blank	0.000	23:35:45
Wait	ABT*	
Air Blank	0.000	23:36:20

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

02/01/2018

23:36:26

---

Standard Lot#: 06117100A2

Last Changed By: B. PENNER

#29

---

QAS: A. LAWRENCE  
PRESCOTT VALLEY PD

#61

---

Operator: A. LAWRENCE  
PRESCOTT VALLEY PD

#61

---

Subject: SQAP, TEST

DOB: 01/05/1980

Sex: M

Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	23:37:10
Air Blank	0.000	23:37:39

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

02/01/2018

23:37:43

---

Standard Lot#: 06117100A2

Last Changed By: B. PENNER

#29

---

QAS: A. LAWRENCE  
PRESCOTT VALLEY PD

#61

---

Operator: A. LAWRENCE  
PRESCOTT VALLEY PD

#61

---

Subject: SQAP, TEST

DOB: 01/05/1980

Sex: M

Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	23:38:27
Air Blank	0.000	23:38:55

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT VALLEY PD  
Serial Number: 80-000293  
Core Version: 8105.48  
02/01/2018

23:38:59

Standard Lot#: 06117100A2  
Last Changed By: B. PENNER #29

QAS: A. LAWRENCE #61  
PRESCOTT VALLEY PD

Operator: A. LAWRENCE #61  
PRESCOTT VALLEY PD

Subject: SQAP, TEST  
DOB: 01/05/1980  
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:39:47
Diagnostic Test	Pass	23:40:21
Air Blank	0.000	23:40:49
0.100 Cal Check	0.099	23:41:09
Air Blank	0.000	23:41:39
Subject Test	0.162*	23:44:46
Air Blank	0.000	23:45:18
Air Blank	0.000	23:45:46
Subject Test	XXX**	23:46:19
Air Blank	0.000	23:46:53

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. LAWRENCE AGENCY PRESOTT VALLEY PD  
DATE 2-1-18 TIME 2347  
INTOXILYZER SERIAL # 80-000293 LOCATION PVPD/ BOOKING ROOM

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE \_\_\_\_\_

DPS Form Exh G-2 (Rev 05-01)

INTOXILYZER 8000

Location: PRESCOTT VALLEY PD

Serial Number: 80-000293

Core Version: 8105.48

02/01/2018

23:47:08

-----  
Standard Lot#: 06117100A2

Last Changed By: B. PENNER #29  
-----

QAS: A. LAWRENCE #61

PRESCOTT VALLEY PD  
-----

Operator: A. LAWRENCE #61

PRESCOTT VALLEY PD  
-----



Test	g/210L	Time
Air Blank	0.000	23:47:38
0.100 Cal Check	0.098	23:47:59
Air Blank	0.000	23:48:29

Preventative Maintenance  
Performed -- Timer Reset