

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. AMENBERGER #250 AGENCY Prescott P.D.  
DATE 04/008 TIME 2335  
INTOXILYZER SERIAL # 80-000337 LOCATION Prescott P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
~~Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(✓) 4. Air blank completed.  
(✓) 5. Calibration check completed. Test results 0. 100 AC.  
(✓) 6. Air blank completed.  
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE T. Amenberger #250

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

04/10/2008

23:35:29

-----  
Standard Lot#: 727501I

Last Changed By: T. ATTENBERGER #252

-----  
QAS: T. ATTENBERGER #252

PRESCOTT PD  
-----

Test	g/210L	Time
Air Blank	0.000	23:35:56
Cal Check	0.100	23:36:15
Air Blank	0.000	23:36:45

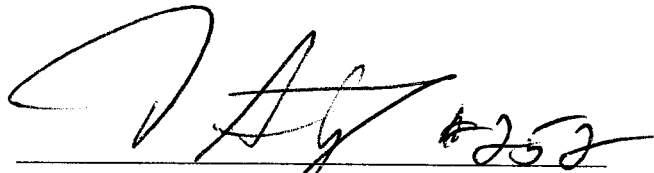
  
-----  
Operator's Signature

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST T. Anderson #357 AGENCY Prescott P.D.  
DATE 04/10/08 TIME 2339  
INTOXILYZER SERIAL # 80-000337 LOCATION P.P.D.

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

ALL OPERATING CORRECTLY.

SIGNATURE [Signature] #357

INTOXILYZER 8000  
Location: PRESCOTT P.D  
Serial Number : 80-000337  
04/10/2008

23:39:01

---

Standard Lot#: 727501I  
Last Changed By: T. ATTENBERGER #252

---

QAS: T. ATTENBERGER #252  
PRESCOTT PD

---

Operator: T. ATTENBERGER #252  
PRESCOTT PD

---

Subject: SQAP, TEST  
DOB: 01/01/1900  
Sex: M Weight: 300

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	23:40:44
Diagnostic Test	Pass	23:41:12
Air Blank	0.000	23:41:39
0.100 Cal Check	0.099	23:41:58
Air Blank	0.000	23:42:27
Subject Test	0.000	23:42:43
Air Blank	0.000	23:43:10
Five Minute Wait	ABT*	
Air Blank	0.000	23:43:46

\*Sequence Aborted

A handwritten signature in black ink, appearing to be "T. Attenberger #252".

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT P.D  
Serial Number : 80-000337  
04/10/2008 23:44:42

---

Standard Lot#: 727501I  
Last Changed By: T. ATTENBERGER #252

---

QAS: T. ATTENBERGER #252  
PRESCOTT PD

---

Operator: T. ATTENBERGER #252  
PRESCOTT PD

---

Subject: SQAP, TEST  
DOB: 01/01/1900  
Sex: M Weight: 300

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	23:45:38
Air Blank	0.000	23:46:06

\*Improper Sample



**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT P.D  
Serial Number : 80-000337  
04/10/2008 23:46:46

---

Standard Lot#: 727501I  
Last Changed By: T. ATTENBERGER #252

---

QAS: T. ATTENBERGER #252  
PRESCOTT PD

---

Operator: T. ATTENBERGER #252  
PRESCOTT PD

---

Subject: SQAP,TEST  
DOB: 01/01/1900  
Sex: M Weight: 300


---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	23:47:37
Air Blank	0.000	23:48:05

\*RFI Detect

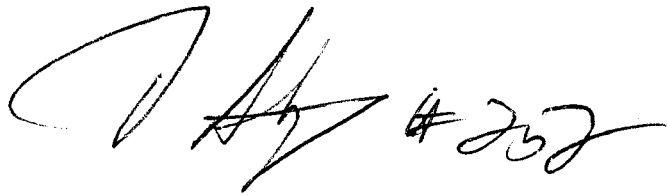

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT P.D  
Serial Number : 80-000337  
04/10/2008 23:48:53

---

Standard Lot#: 727501I  
Last Changed By: T. ATTENBERGER #252

---

QAS: T. ATTENBERGER #252  
PRESCOTT PD

---

Operator: T. ATTENBERGER #252  
PRESCOTT PD

---

Subject: SQAP, TEST  
DOB: 01/01/1900  
Sex: M Weight: 300

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	23:50:07
Diagnostic Test	Pass	23:50:34
Air Blank	0.000	23:51:02
0.100 Cal Check	0.098	23:51:20
Air Blank	0.000	23:51:50
Subject Test	0.040*	23:54:58
Air Blank	0.000	23:55:29
Air Blank	0.000	23:55:57
Subject Test	XXX**	23:56:24
Air Blank	0.000	23:56:56

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period



**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. Arreaga #250 AGENCY Prescott P.D.  
DATE 04/008 TIME 2358  
INTOXILYZER SERIAL # 80-000337 LOCATION P.P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(✓) 4. Air blank completed.  
(✓) 5. Calibration check completed. Test results 0. 098 AC.  
(✓) 6. Air blank completed.  
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE T. Arreaga #250

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

04/10/2008

23:58:45


-----  
Standard Lot#: 727501I

Last Changed By: T. ATTENBERGER #252

-----  
QAS: T. ATTENBERGER #252

PRESCOTT PD  
-----

Test	g/210L	Time
Air Blank	0.000	23:59:12
Cal Check	0.098	23:59:31
Air Blank	0.000	00:00:00

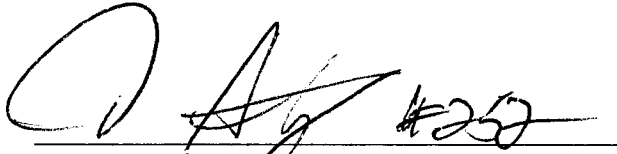  
-----  
Operator's Signature