

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

04/20/2008

02:14:24


Standard Lot#: 727501I

Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252

PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	02:14:51
Cal Check	0.099	02:15:10
Air Blank	0.000	02:15:40

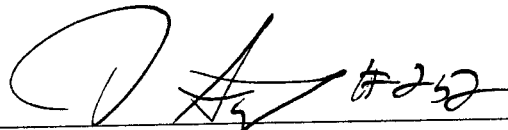

Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST T. AMENBERGER #20 AGENCY Prescott P.D.
DATE 04/20/08 TIME 0216
INTOXILYZER SERIAL # 80-000337 LOCATION Prescott P.D.

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.


OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

I BLEW TOO LONG ON "DEFICIENT SAMPLE". "MOUTH ALCOHOL"
PERFORMED (CORRECT); "DEFICIENT SAMPLE" & "MOUTH ALCOHOL"
PERFORMED A SECOND TIME.
ALL FUNCTIONING CORRECTLY.

SIGNATURE  #20

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number : 80-000337
04/20/2008 02:16:40

Standard Lot#: 727501I
Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP, TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	02:17:50
Diagnostic Test	Pass	02:18:18
Air Blank	0.000	02:18:45
0.100 Cal Check	0.098	02:19:04
Air Blank	0.000	02:19:33
Subject Test	0.000	02:19:58
Air Blank	0.000	02:20:25
Five Minute Wait	ABT*	
Air Blank	0.000	02:21:01

*Sequence Aborted



Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number : 80-000337
04/20/2008 02:22:40

Standard Lot#: 727501I
Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252
PRESCOTT PD


Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP,TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	02:23:33
Air Blank	0.000	02:24:01

*Improper Sample

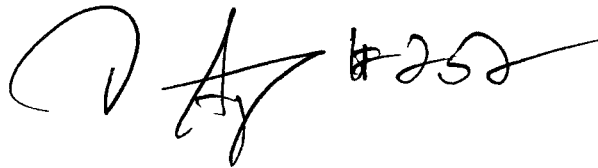

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number : 80-000337
04/20/2008 02:25:43

Standard Lot#: 727501I
Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP, TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	02:27:01
Air Blank	0.000	02:27:29

*RFI Detect



Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number : 80-000337
04/20/2008 02:29:08

Standard Lot#: 727501I
Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP, TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	02:30:08
Diagnostic Test	Pass	02:30:35
Air Blank	0.000	02:31:03
0.100 Cal Check	0.099	02:31:21
Air Blank	0.000	02:31:51
Subject Test	0.009	02:32:31
Air Blank	0.000	02:32:58
Five Minute Wait		
Air Blank	0.000	02:37:59
Subject Test	XXX*	02:38:22
Air Blank	0.000	02:38:57

*Invalid Sample
Begin new deprivation period

I BLEW TOO LONG ON "DEFICIENT SAMPLE".
"MOVEH A LOW" CHECK OPERATIONS CORRECTLY.
ABOVE 2 REPEATS SO A SECOND TIME (SEE NEXT
PAGE).

Not a Successfully Completed Test Sequence

 #252

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number : 80-000337
04/20/2008

02:42:34

Standard Lot#: 727501I
Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP,TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	02:43:38
Diagnostic Test	Pass	02:44:05
Air Blank	0.000	02:44:33
0.100 Cal Check	0.099	02:44:51
Air Blank	0.000	02:45:21
Subject Test	0.009*	02:48:29
Air Blank	0.000	02:48:59
Air Blank	0.000	02:49:26
Subject Test	XXX**	02:49:49
Air Blank	0.000	02:50:23

*Deficient Sample

**Invalid Sample

Begin new deprivation period



Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. ARONBERG #252 AGENCY Prescott P.D.
DATE 07/20/08 TIME 0252
INTOXILYZER SERIAL # 80-000337 LOCATION Prescott P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.150 AC.
OR
Pour a standard alcohol concentration solution 0.150 AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 099 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove ~~printed~~ record. Attach the record to the completed checklist.

SIGNATURE [Signature] #252

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

04/20/2008

02:52:30

Standard Lot#: 727501I

Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252

PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	02:52:58
Cal Check	0.099	02:53:17
Air Blank	0.000	02:53:46



Operator's Signature