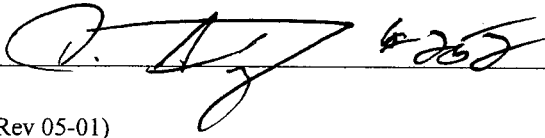


EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. ADENBERGER #252 AGENCY Prescott P.D.  
DATE 051508 TIME 1913  
INTOXILYZER SERIAL # 80-000337 LOCATION P.P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select (D) for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  #252

DPS Form Exh G-2 (Rev 05-01)

\* PERFORMED G2/G4/G9 AS BATTERY BACK UP TO BE REPAIRED. (POWERED/NEOX. OFF/ON)

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

05/15/2008

19:13:37

-----  
Standard Lot#: 727501I

Last Changed By: T. ATTENBERGER #252

-----  
QAS: T. ATTENBERGER #252

PRESCOTT PD  
-----

Test	g/210L	Time
Air Blank	0.000	19:14:04
Cal Check	0.099	19:14:23
Air Blank	0.000	19:14:53

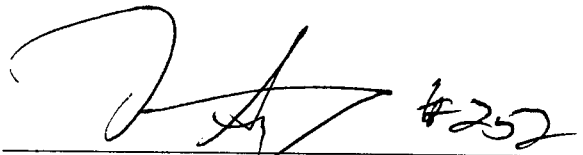
  
-----  
Operator's Signature

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST T. ARZUMBARCAN #257 AGENCY PRESCOTT P.D.  
DATE 051508 TIME 1918  
INTOXILYZER SERIAL # 80-000337 LOCATION P.P.D.

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.


Instrument is operating properly and accurately. YES  NO

COMMENTS

ALL FUNCTIONING PROPERLY.

\* G2/G4/G2 REFORMED AS POWERSO (NEOX OFF/ON).

SIGNATURE

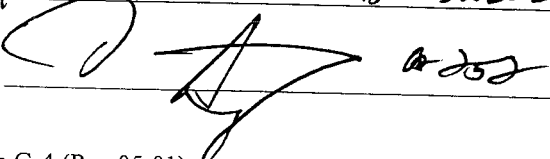

INTOXILYZER 8000  
Location: PRESCOTT P.D  
Serial Number : 80-000337  
05/15/2008

19:18:13

---

Standard Lot#: 727501I  
Last Changed By: T. ATTENBERGER #252

---

QAS: T. ATTENBERGER #252  
PRESCOTT PD

---

Operator: T. ATTENBERGER #252  
PRESCOTT PD

---

Subject: SQAP, TEST  
DOB: 01/01/1900  
Sex: M Weight: 300

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	19:19:39
Diagnostic Test	Pass	19:20:07
Air Blank	0.000	19:20:34
0.100 Cal Check	0.098	19:20:53
Air Blank	0.000	19:21:22
Subject Test	0.000	19:21:37
Air Blank	0.000	19:22:05
Five Minute Wait	ABT*	
Air Blank	0.000	19:22:40

\*Sequence Aborted



**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT P.D  
Serial Number : 80-000337  
05/15/2008

19:23:26

---

Standard Lot#: 727501I  
Last Changed By: T. ATTENBERGER #252

---

QAS: T. ATTENBERGER #252  
PRESCOTT PD

---

Operator: T. ATTENBERGER #252  
PRESCOTT PD

---

Subject: SQAP,TEST  
DOB: 01/01/1900  
Sex: M Weight: 300

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	19:24:13
Air Blank	0.000	19:24:41

\*Improper Sample

*[Handwritten signature]* #252

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT P.D  
Serial Number : 80-000337  
05/15/2008

19:25:15

---

Standard Lot#: 727501I  
Last Changed By: T. ATTENBERGER #252

---

QAS: T. ATTENBERGER #252  
PRESCOTT PD

---

Operator: T. ATTENBERGER #252  
PRESCOTT PD

---

Subject: SQAP, TEST  
DOB: 01/01/1900  
Sex: M Weight: 300

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	19:26:12
Air Blank	0.000	19:26:40

\*RFI Detect

 #252

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT P.D  
Serial Number : 80-000337  
05/15/2008

19:27:21

---

Standard Lot#: 727501I  
Last Changed By: T. ATTENBERGER #252

---

QAS: T. ATTENBERGER #252  
PRESCOTT PD

---

Operator: T. ATTENBERGER #252  
PRESCOTT PD

---

Subject: SQAP, TEST  
DOB: 01/01/1900  
Sex: M Weight: 300

---

15 Minute Deprivation Period? Yes


---

Test	g/210L	Time
Air Blank	0.000	19:28:20
Diagnostic Test	Pass	19:28:48
Air Blank	0.000	19:29:15
0.100 Cal Check	0.098	19:29:34
Air Blank	0.000	19:30:03
Subject Test	0.009*	19:33:12
Air Blank	0.000	19:33:44
Air Blank	0.000	19:34:11
Subject Test	XXX**	19:34:37
Air Blank	0.000	19:35:11

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

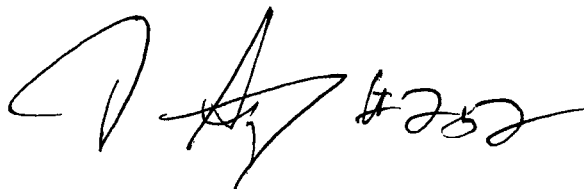

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. Amberg AGENCY P. P. D.  
DATE 051508 TIME 1938  
INTOXILYZER SERIAL # 80-000337 LOCATION P.P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(✓) 3. Go to the "Control Testing Menu". Select (D) for dry control test or "W" for wet control test. After selection is made press ENTER.  
(✓) 4. Air blank completed.  
(✓) 5. Calibration check completed. Test results 0. 097 AC.  
(✓) 6. Air blank completed.  
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE T. Amberg

DPS Form Exh G-2 (Rev 05-01)


\* Performed G2/G4/G2 AS POWERED WEDX  
05-15-08.

INTOXILYZER 8000  
Location: PRESCOTT P.D  
Serial Number: 80-000337  
05/15/2008 19:38:06

-----  
Standard Lot#: 727501I  
Last Changed By: T. ATTENBERGER #252  
-----

QAS: T. ATTENBERGER #252  
PRESCOTT PD  
-----

Test	g/210L	Time
Air Blank	0.000	19:38:34
Cal Check	0.097	19:38:52
Air Blank	0.000	19:39:22

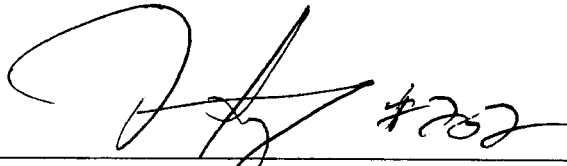

Operator's Signature