

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

05/06/2011

11:31:18

Standard Lot#: 25310100A1

Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252

PRESCOTT PD

Operator: T. ATTENBERGER #252

PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	11:31:50
0.100 Cal Check	0.100	11:32:10
Air Blank	0.000	11:32:40

Preventative Maintenance
Performed -- Timer Reset



EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST T. Amador 6252 AGENCY Prescott P.D.
DATE 050601 TIME 1136
INTOXILYZER SERIAL # 80-000337 LOCATION P.P.D.

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

ALL FUNCTIONING CORRECTLY.

SIGNATURE [Signature]

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number: 80-000337
Core Version: 8105.48
05/06/2011 11:36:56

Standard Lot#: 25310100A1
Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP,TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:38:04
Diagnostic Test	Pass	11:38:38
Air Blank	0.000	11:39:06
0.100 Cal Check	0.100	11:39:26
Air Blank	0.000	11:39:56
Subject Test	0.000	11:40:16
Air Blank	0.000	11:40:44
Wait	ABT*	
Air Blank	0.000	11:41:21

*Sequence Aborted



Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

05/06/2011

11:41:49

Standard Lot#: 25310100A1

Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252

PRESCOTT PD

Operator: T. ATTENBERGER #252

PRESCOTT PD

Subject: SQAP, TEST

DOB: 01/01/1900

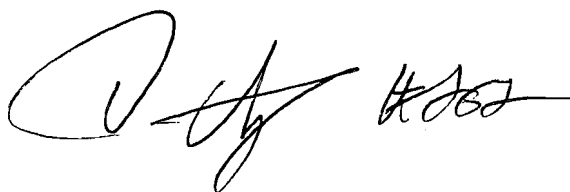
Sex: M

Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	11:42:27
Air Blank	0.000	11:42:55

*Improper Sample

A handwritten signature in black ink, appearing to be 'T. Attenberger' with a large initial 'T' and 'A'.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number: 80-000337
Core Version: 8105.48
05/06/2011 11:43:39

Standard Lot#: 25310100A1
Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP,TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	11:44:27
Air Blank	0.000	11:44:55

*RFI Detect

A handwritten signature in black ink, appearing to be 'T. Attenberger', is written over the page.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

05/06/2011

11:45:43

Standard Lot#: 25310100A1

Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252

PRESCOTT PD

Operator: T. ATTENBERGER #252

PRESCOTT PD

Subject: SQAP,TEST

DOB: 01/01/1900

Sex: M

Weight: 300

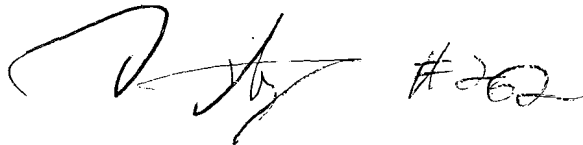
15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:46:45
Diagnostic Test	Pass	11:47:19
Air Blank	0.000	11:47:46
0.100 Cal Check	0.099	11:48:07
Air Blank	0.000	11:48:37
Subject Test	0.021*	11:51:44
Air Blank	0.000	11:52:14
Air Blank	0.000	11:52:41
Subject Test	XXX**	11:53:28
Air Blank	0.000	11:54:01

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Handwritten signature and initials, possibly "T. Attenberger #252".

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. AMANMAGNAN 10252 AGENCY PLAZA P.D.
DATE 050611 TIME 1154
INTOXILYZER SERIAL # 80-000737 LOCATION P.O.D.

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 098 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

05/06/2011

11:54:52

Standard Lot#: 25310100A1

Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252

PRESCOTT PD

Operator: T. ATTENBERGER #252

PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	11:55:42
0.100 Cal Check	0.098	11:56:03
Air Blank	0.000	11:56:33

Preventative Maintenance

Performed -- Timer Reset

A handwritten signature in black ink, appearing to be 'T. Attenberger', is written across the bottom of the page.