

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. A. [Signature] #252 AGENCY Phoenix P.D.
DATE 051711 TIME 0825
INTOXILYZER SERIAL # 80-000387 LOCATION PPD

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 100 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

* SHUT DOWN INTOX 8000 TO REPLACE "APC" METER
BACK-UP UNIT. NO ISSUES WITH INTOX 8000. G-2
FROM TO TURNING ON INTOX 8000.

[Signature]

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

05/17/2011

08:25:18

Standard Lot#: 25310100A1

Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252

PRESCOTT PD

Operator: T. ATTENBERGER #252

PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	08:25:48
0.100 Cal Check	0.100	08:26:09
Air Blank	0.000	08:26:38

Preventative Maintenance
Performed -- Timer Reset

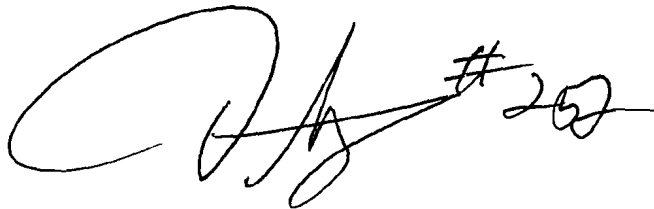
A handwritten signature in black ink, appearing to be 'T. Attenberger', followed by the number '#252' written to the right of the signature.

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. ARABERGER #252 AGENCY PRISON PD
DATE 051711 TIME 1341
INTOXILYZER SERIAL # 80-050337 LOCATION PRD

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 099 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE T. Araberger #252

DPS Form Exh G-2 (Rev 05-01)

* G-2 UPON TURNING POWER TO INTOX 8000 BACK ON. NO ISSUE W/ INTOX 8000. POWERED BACK ON AFTER REPLACING BATTERY IN "APC" BATTERY BACK UP UNIT.

T. Araberger #252

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number: 80-000337
Core Version: 8105.48
05/17/2011 13:41:05

Standard Lot#: 25310100A1
Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	13:41:38
0.100 Cal Check	0.099	13:41:59
Air Blank	0.000	13:42:28

Preventative Maintenance
Performed -- Timer Reset

A handwritten signature in black ink, appearing to be 'T. Attenberger #252', written in a cursive style.