

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. A. [Signature] #252 AGENCY Peisong P.D.
DATE 080611 TIME 0844
INTOXILYZER SERIAL # 80-000337 LOCATION PPD.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution. 100 AC.
OR
Pour a standard alcohol concentration solution AC into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 097 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 252

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number: 80-000337
Core Version: 8105.48
08/26/2011 08:41:13

Standard Lot#: 25310100A1
Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	08:41:45
0.100 Cal Check	0.097	08:42:06
Air Blank	0.000	08:42:36

Preventative Maintenance
Performed -- Timer Reset

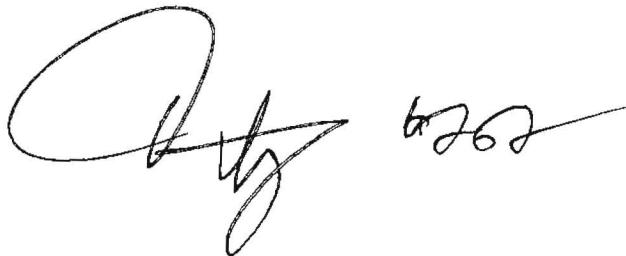


EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST T. Anderson #150 AGENCY Prison P.D.
DATE 082611 TIME 0844
INTOXILYZER SERIAL # 80-000337 LOCATION PRD

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

ALL FUNCTIONING CORRECTLY

SIGNATURE [Signature]

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number: 80-000337
Core Version: 8105.48
08/26/2011 08:44:13

Standard Lot#: 25310100A1
Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP,TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:45:14
Diagnostic Test	Pass	08:45:48
Air Blank	0.000	08:46:15
0.100 Cal Check	0.097	08:46:36
Air Blank	0.000	08:47:06
Subject Test	0.000	08:47:24
Air Blank	0.000	08:47:51
Wait	ABT*	
Air Blank	0.000	08:48:27

*Sequence Aborted

A handwritten signature in black ink, appearing to be 'T. Attenberger', with a stylized flourish at the end.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

08/26/2011

09:06:22

Standard Lot#: 25310100A1

Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252

PRESCOTT PD

Operator: T. ATTENBERGER #252

PRESCOTT PD

Subject: SQAP, TEST

DOB: 01/01/1900

Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	09:07:36
Air Blank	0.000	09:08:05

*Improper Sample

A handwritten signature in black ink, appearing to be 'T. Attenberger', written in a cursive style.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number: 80-000337
Core Version: 8105.48
08/26/2011 09:08:25

Standard Lot#: 25310100A1
Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252
PRESCOTT PD


Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP,TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	09:09:26
Air Blank	0.000	09:09:54

*RFI Detect

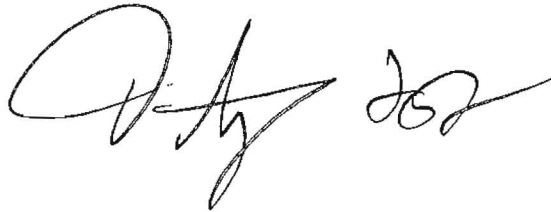A handwritten signature in black ink, appearing to be 'D. Ay' followed by a flourish.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

08/26/2011

09:10:42

Standard Lot#: 25310100A1

Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252

PRESCOTT PD

Operator: T. ATTENBERGER #252

PRESCOTT PD

Subject: SQAP,TEST

DOB: 01/01/1900

Sex: M

Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:11:35
Diagnostic Test	Pass	09:12:09
Air Blank	0.000	09:12:36
0.100 Cal Check	0.097	09:12:57
Air Blank	0.000	09:13:26
Subject Test	0.011*	09:16:34
Air Blank	0.000	09:17:04
Air Blank	0.000	09:17:32
Subject Test	XXX**	09:18:08
Air Blank	0.000	09:18:41

*Deficient Sample

**Invalid Sample

Begin new deprivation period



Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. KYLE BURGESS #252 AGENCY Prescott P.D.
DATE 082611 TIME 0919
INTOXILYZER SERIAL # 80-000337 LOCATION PPD

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .600 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.097 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

08/26/2011

09:19:29

Standard Lot#: 25310100A1

Last Changed By: T. ATTENBERGER #252

QAS: T. ATTENBERGER #252

PRESCOTT PD

Operator: T. ATTENBERGER #252

PRESCOTT PD

Test g/210L Time

Air Blank 0.000 09:20:00
0.100 Cal Check 0.097 09:20:21
Air Blank 0.000 09:20:50

Preventative Maintenance
Performed -- Timer Reset

