

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. Amador #252 AGENCY Prison PD
DATE 030712 TIME 1544
INTOXILYZER SERIAL # 80-000332 LOCATION PPD

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$~~
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
() 3. Go to the "Control Testing Menu". Select D for dry control test or "W" for wet control test. After selection is made press ENTER.
() 4. Air blank completed.
() 5. Calibration check completed. Test results 0.089 AC.
() 6. Air blank completed.
() 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

ZELLEN SPIT LARGE AMOUNT OF SALIVA INTO BREATH TUBE ON PRIOR BREATH TEST CAUSING AIRGE FALL & CAL ✓ OUT OF TOLERANCE (82). SALIVA WAS DRIPPING OUT OF END OF TUBE. MIKE FROM DPS CRIME LAB CONFIRMED THIS. 2nd G-2 WAS W/LN TOLERANCE & SUCCESSFULLY COMPLETED BREATH TEST FOLLOWING TUBE.

[Signature]

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

03/07/2012

15:44:57

Standard Lot#: 17911100A1

Last Changed By: M. MEDINA #356

QAS: M. MEDINA #356

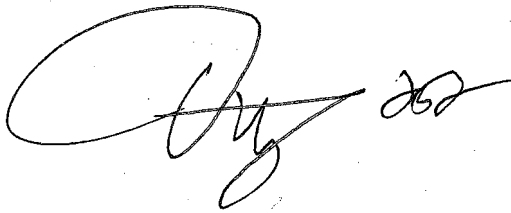
PRESCOTT PD

Operator: T. ATTENBERGER #252

PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	15:45:32
0.100 Cal Check	0.089*	15:45:53
Air Blank	0.000	15:46:28

*Calibration Check Out of Tolerance

A handwritten signature in black ink, appearing to be 'T. Attenberger', with a date '03/07' written to the right of the signature.