

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MEDINA AGENCY PRESCOTT P.D.
DATE 10/12/2012 TIME 0745
INTOXILYZER SERIAL # 80-000337 LOCATION PRESCOTT P.D.

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- * () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START" CONTACT GAS
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 100 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

10/12/12
[Signature]

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number: 80-000337
Core Version: 8105.48
10/12/2012 07:45:00

Standard Lot#: 09312100A1
Last Changed By: T. ATTENBERGER #252

QAS: M. MEDINA #356
PRESCOTT PD

Operator: M. MEDINA #356
PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	07:45:39
0.100 Cal Check	0.100	07:45:59
Air Blank	0.000	07:46:29

Preventative Maintenance
Performed -- Timer Reset

001