

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Todd Anderson #252 AGENCY Prescott PID
DATE 07/25/13 TIME 1849
INTOXILYZER SERIAL # 80-000337 LOCATION P.P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$~~
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select D for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 094 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

07/25/2013

18:49:32

Standard Lot#: 34612100A1

Last Changed By: M. MEDINA #356

QAS: T. ATTENBERGER #252

PRESCOTT PD

Operator: T. ATTENBERGER #252

PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	18:50:05
0.100 Cal Check	0.094	18:50:25
Air Blank	0.000	18:50:55

Preventative Maintenance
Performed -- Timer Reset

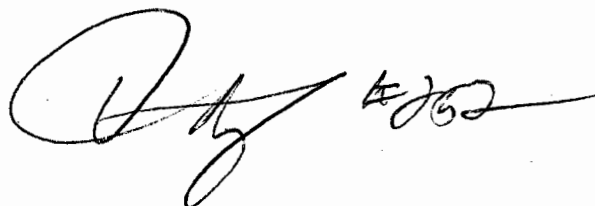
A handwritten signature in black ink, appearing to read 'T. Attenberger', with a stylized flourish extending to the right.

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Anderson #252 AGENCY Prescott P.D.
DATE 02/25/13 TIME 1854
INTOXILYZER SERIAL # 80-000337 LOCATION P.P.D.

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check,

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 094 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

ALL FUNCTIONING CORRECTLY.

SIGNATURE D. Anderson #252

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number: 80-000337
Core Version: 8105.48
07/25/2013

18:54:20

Standard Lot#: 34612100A1
Last Changed By: M. MEDINA #356

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP, TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:55:42
Diagnostic Test	Pass	18:56:16
Air Blank	0.000	18:56:44
0.100 Cal Check	0.095	18:57:04
Air Blank	0.000	18:57:34
Subject Test	0.000	18:58:02
Air Blank	0.000	18:58:29
Wait	ABT*	
Air Blank	0.000	18:59:06

*Sequence Aborted



Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

07/25/2013

19:00:17

Standard Lot#: 34612100A1

Last Changed By: M. MEDINA

#356

QAS: T. ATTENBERGER

#252

PRESCOTT PD

Operator: T. ATTENBERGER

#252

PRESCOTT PD

Subject: SQAP, TEST

DOB: 01/01/1900


Sex: M

Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	19:01:04
Air Blank	0.000	19:01:33

*Improper Sample

A handwritten signature in black ink, appearing to be 'T. Attenberger', is written over the page.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

07/25/2013

19:03:08

Standard Lot#: 34612100A1

Last Changed By: M. MEDINA

#356

QAS: T. ATTENBERGER

#252

PRESCOTT PD

Operator: T. ATTENBERGER

#252

PRESCOTT PD

Subject: SQAP, TEST

DOB: 01/01/1900

Sex: M

Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	19:03:43
Air Blank	0.000	19:04:12

*RFI Detect

A handwritten signature in black ink, appearing to read "T. Attenberger", is written over the page.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

07/25/2013

19:05:14

Standard Lot#: 34612100A1

Last Changed By: M. MEDINA

#356

QAS: T. ATTENBERGER

#252

PRESCOTT PD

Operator: T. ATTENBERGER

#252

PRESCOTT PD

Subject: SQAP, TEST

DOB: 01/01/1900

Sex: M

Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:06:08
Diagnostic Test	Pass	19:06:42
Air Blank	0.000	19:07:10
0.100 Cal Check	0.095	19:07:30
Air Blank	0.000	19:08:00
Subject Test	0.023*	19:11:08
Air Blank	0.000	19:11:38
Air Blank	0.000	19:12:05
Subject Test	XXX**	19:12:43
Air Blank	0.000	19:13:16

*Deficient Sample

**Invalid Sample

Begin new deprivation period

A handwritten signature in black ink, appearing to be "T. Attenberger #252".

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST *T. Anderson* AGENCY *P. P. D.*
DATE *07/25/13* TIME *1913*
INTOXILYZER SERIAL # *80-00737* LOCATION *P.P.D.*

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution *1.00* AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "*D*" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results *0.014* AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____
T. Anderson

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

07/25/2013

19:13:55

Standard Lot#: 34612100A1

Last Changed By: M. MEDINA #356

QAS: T. ATTENBERGER #252

PRESCOTT PD

Operator: T. ATTENBERGER #252

PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	19:14:26
0.100 Cal Check	0.094	19:14:47
Air Blank	0.000	19:15:17

Preventative Maintenance
Performed -- Timer Reset

