

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000


STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST S. Hester AGENCY Prescott P.D.

DATE 3/6/17 TIME 1504

INTOXILYZER SERIAL # 80-000337 LOCATION 222 S Main St

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

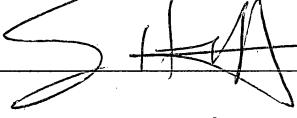SIGNATURE 

DPS Form Exh G-2 (Rev 05-01)

Prescott PD CHANGE 5-12-06

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

03/06/2017

15:06:25

Standard Lot#: 22315100A1

Last Changed By: S. HILTON #372

QAS: S. HILTON #372

PRESCOTT PD

Operator: S. HILTON #372

PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	15:06:56
0.100 Cal Check	0.098	15:07:17
Air Blank	0.000	15:07:47

Preventative Maintenance
Performed -- Timer Reset

SN

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST S. Helton AGENCY Prescott P.D.
DATE 3/6/17 TIME 1504
INTOXILYZER SERIAL # 80-000337 LOCATION 222 S. MARINA ST.

4/5 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 4/5 1. Clock time check.
4/5 2. Date check.

OPERATIONAL TESTS

- 4/5 1. Alcohol-free subject test result 0. 000 AC.
4/5 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
4/5 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
4/5 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

Functioning Properly
* PLACED INTO SERVICE AS 006230
IS DUE FOR ANNUAL SERVICE @ DPS

SIGNATURE S. Helton

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number: 80-000337
Core Version: 8105.48
03/06/2017

15:10:02

Standard Lot#: 22315100A1
Last Changed By: S. HILTON #372

QAS: S. HILTON #372
PRESCOTT PD

Operator: S. HILTON #372
PRESCOTT PD

Subject: SQAP, TEST
DOB: 12/19/1973
Sex: M Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:10:54
Diagnostic Test	Pass	15:11:28
Air Blank	0.000	15:11:56
0.100 Cal Check	0.098	15:12:17
Air Blank	0.000	15:12:47
Subject Test	0.000	15:13:02
Air Blank	ABT*	15:13:29
Air Blank	0.000	15:13:58

*Sequence Aborted



Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

03/06/2017

15:18:21

Standard Lot#: 22315100A1

Last Changed By: S. HILTON

#372

QAS: S. HILTON

#372

PRESCOTT PD

Operator: S. HILTON

#372

PRESCOTT PD

Subject: SQAP,TEST

DOB: 12/19/1973


Sex: M

Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	15:19:12
Air Blank	0.000	15:19:40

*Improper Sample

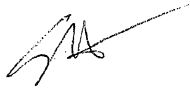

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

03/06/2017

15:20:10

Standard Lot#: 22315100A1

Last Changed By: S. HILTON

#372

QAS: S. HILTON

#372

PRESCOTT PD

Operator: S. HILTON

#372

PRESCOTT PD

Subject: SQAP, TEST

DOB: 12/19/1973

Sex: M

Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	15:20:53
Air Blank	0.000	15:21:21

*RFI Detect



Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT P.D
Serial Number: 80-000337
Core Version: 8105.48
03/06/2017

15:21:39

Standard Lot#: 22315100A1
Last Changed By: S. HILTON #372

QAS: S. HILTON #372
PRESCOTT PD

Operator: S. HILTON #372
PRESCOTT PD

Subject: SQAP, TEST
DOB: 12/19/1973
Sex: M Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:22:33
Diagnostic Test	Pass	15:23:07
Air Blank	0.000	15:23:34
0.100 Cal Check	0.095	15:23:55
Air Blank	0.000	15:24:25
Subject Test	0.199*	15:27:33
Air Blank	0.000	15:28:15
Air Blank	0.000	15:28:43
Subject Test	XXX**	15:29:06
Air Blank	0.000	15:30:03

*Deficient Sample

**Invalid Sample

Begin new deprivation period



Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST S. Hill AGENCY Prescott P.P.
DATE 3/6/17 TIME 1530
INTOXILYZER SERIAL # 80-000337 LOCATION 222 S. MARINA ST.

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 097 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE S Hill

Prescott PID CHANGE 5-18-06

INTOXILYZER 8000

Location: PRESCOTT P.D

Serial Number: 80-000337

Core Version: 8105.48

03/06/2017

15:30:26

Standard Lot#: 22315100A1

Last Changed By: S. HILTON #372

QAS: S. HILTON #372

PRESCOTT PD

Operator: S. HILTON #372

PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	15:30:57
0.100 Cal Check	0.097	15:31:18
Air Blank	0.000	15:31:48

Preventative Maintenance
Performed -- Timer Reset

SW