

**COPY**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST S. Blakley AGENCY Yavapai County Sheriff's Office  
DATE 12-14-15 TIME 14:23:07  
INTOXILYZER SERIAL # 80-006228 LOCATION DUI Truck

- ( ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution — AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (~~NO~~) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(~~✓~~) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(~~✓~~) 4. Air blank completed.  
(~~✓~~) 5. Calibration check completed. Test results 0.100 AC.  
(~~✓~~) 6. Air blank completed.  
(~~✓~~) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] LESA/W42

DPS Form Exh G-2 (Rev 05-01)

Tank # 01314100A3  
Exp 02-01-16

INTOXILYZER 8000  
Instrument Initialization  
13:33:57 12/14/2015

INTOXILYZER 8000  
Location: YAUPPAI COUNTY SO  
Serial Number: 80-006228  
Core Version: B105.48  
12/14/2015 14:23:07

-----  
Standard Lot#: 0131410023  
Last Changed By: J. MURRAY5557 #N56  
-----

QAS: S. BLAKLEY SR #L6861  
YAUPPAI CSC  
-----

Operator: S. BLAKLEY SR #L6861  
YAUPPAI CSC  
-----

Test	g/210L	Time
Air Blank	0.000	14:23:59
0.100 Cal Check	0.100	14:24:20
Air Blank	0.000	14:24:49

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST S. Blakley AGENCY Yavapai County Sheriff's Office  
DATE 12-14-15 TIME 14:33:20  
INTOXILYZER SERIAL # 30-006-228 LOCATION DUI Truck

- ( ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.10% AC.  
OR  
Pour a standard alcohol concentration solution --- AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (x) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (x) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (x) 4. Air blank completed.
- (x) 5. Calibration check completed. Test results 0.09% AC.
- (x) 6. Air blank completed.
- (x) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] LLS/1/142

DPS Form Exh G-2 (Rev 05-01)

Tank # 0131410093  
Exp 02-01-16

