

DR 17-011300

Copy

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST S. Blakley Sr. AGENCY Yavapai County SO

DATE 4-1-17 TIME 00:18:22

INTOXILYZER SERIAL # 80-006228 LOCATION DUI Truck YCSO

- (x) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (x) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (x) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (x) 4. Air blank completed.
- (x) 5. Calibration check completed. Test results 0. 099 AC.
- (x) 6. Air blank completed.
- (x) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

Tank Serial # 15615100A1

Exp 7-5-2017

press 800 psi

RECEIVED

APR 04 2017

YAVAPAI COUNTY ATTORNEY

DR 17-011308

INTOXILYZER 8000
Location: YAVAPAI COUNTY SO
Serial Number: 80-006228
Core Version: 8105.48
04/01/2017 00:12:57

Standard Lot#: 15615100A1
Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	00:13:27
0.100 Cal Check	0.099	00:13:48
Air Blank	0.000	00:14:18

Preventative Maintenance
Performed -- Timer Reset

Instrument/Vehicle out of area
Past 31 day mark

DR 17-011300

INTOXILYZER 8000
Location: YAVAPAI COUNTY SO
Serial Number: 80-006228
Core Version: 8105.48
04/01/2017 00:18:22

Standard Lot#: 15615100A1
Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	00:18:56
0.100 Cal Check	0.099	00:19:17
Air Blank	0.000	00:19:46

Preventative Maintenance
Performed -- Timer Reset

AR 17-011300

Copy

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST S. Blakley, Sr. AGENCY Yavapai County SO
DATE 4-1-17 TIME 00:22:08
INTOXILYZER SERIAL # 80-006228 LOCATION DUI Truck YESO

(x) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (x) 1. Clock time check.
- (x) 2. Date check.

OPERATIONAL TESTS

- (x) 1. Alcohol-free subject test result 0. 000 AC.
- (x) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (x) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (x) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES X NO

COMMENTS

SIGNATURE [Signature]

Tank Serial # 15615100A1
Exp. 7-5-2017
press 800 psi

DR 17-011302

INTOXILYZER 8000
Location: YAVAPAI COUNTY SO
Serial Number: 80-006228
Core Version: 8105.48
04/01/2017 00:22:08

Standard Lot#: 15615100A1
Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Subject: SQAP, TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	00:23:27
Diagnostic Test	Pass	00:24:02
Air Blank	0.000	00:24:29
0.100 Cal Check	0.097	00:24:50
Air Blank	0.000	00:25:19
Subject Test	0.000	00:25:38
Air Blank	0.000	00:26:05
Wait	ABT*	
Air Blank	0.000	00:26:43

*Sequence Aborted

Not a Successfully Completed Test Sequence

AR 17-011308

INTOXILYZER 8000
Location: YAVAPAI COUNTY SO
Serial Number: 80-006228
Core Version: 8105.48
04/01/2017 00:29:34

Standard Lot#: 15615100A1
Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Subject: SQAP, TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	00:30:20
Air Blank	RFI**	00:30:41

*Improper Sample
**RFI Detect

Not a Successfully Completed Test Sequence

AR 17-011300

INTOXILYZER 8000
Location: YAVAPAI COUNTY SO
Serial Number: 80-006228
Core Version: 8105.48
04/01/2017 00:31:39

Standard Lot#: 15615100A1
Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	00:32:38
Diagnostic Test	Pass	00:33:12
Air Blank	0.000	00:33:40
0.100 Cal Check	0.098	00:34:01
Air Blank	0.000	00:34:30
Subject Test	0.000*	00:37:37
Air Blank	0.000	00:38:09
Air Blank	0.000	00:38:37
Subject Test	XXX**	00:39:01
Air Blank	0.000	00:39:35

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

DIR 17-071307

INTOXILYZER 8000

Location: YAVAPAI COUNTY SO

Serial Number: 80-006228

Core Version: 8105.48

04/01/2017

00:40:07

Standard Lot#: 15615100A1

Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	00:40:38
0.100 Cal Check	0.097	00:40:58
Air Blank	0.000	00:41:28

Preventative Maintenance

Performed -- Timer Reset

DR 17-011308

Copy

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST S. Blakley Sr. AGENCY Yavapai County SO
DATE 4-1-17 TIME 07:42:32
INTOXILYZER SERIAL # 80-006-228 LOCATION DUI Tank YCSO

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
 Pour a standard alcohol concentration solution ^{OR} NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.097 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

Tank Serial # 15615100A1

Exp 7-5-2017

press 800 psi

DR17-011302

INTOXILYZER 8000

Location: YAVAPAI COUNTY SO

Serial Number: 80-006228

Core Version: 8105.48

04/01/2017

00:42:32

Standard Lot#: 15615100A1

Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	00:43:04
0.100 Cal Check	0.097	00:43:25
Air Blank	0.000	00:43:54

Preventative Maintenance

Performed -- Timer Reset