

DR 18-007631

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scott Blakley Sr. AGENCY Yavapai County SO

DATE 3-2-18 TIME 21:51:34

INTOXILYZER SERIAL # 80-006228 LOCATION YCSO DUI Truck

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

Tank Serial # 06717100A5

Exp 5-5-2019

Press 950 psi

DR 18-007631

INTOXILYZER 8000  
Location: YAVAPAI COUNTY SO  
Serial Number: 80-006228  
Core Version: 8105.48  
03/02/2018 21:51:34

-----  
Standard Lot#: 06717100A5  
Last Changed By: S. BLAKLEY SR #L6861  
-----

QAS: S. BLAKLEY SR #L6861  
YAVAPAI CSO  
-----

Operator: S. BLAKLEY SR #L6861  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	21:52:05
0.100 Cal Check	0.098	21:52:26
Air Blank	0.000	21:52:55

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Scott Blackley Sr. AGENCY Yavapai County SO  
DATE 3-2-18 TIME 21:53:57  
INTOXILYZER SERIAL # 80-006228 LOCATION YCSO DUI Truck

(x) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (x) 1. Clock time check.
- (x) 2. Date check.

OPERATIONAL TESTS

- (x) 1. Alcohol-free subject test result 0. 000 AC.
- (x) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (x) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- ( ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES x NO     

COMMENTS

Instrument Return to Service  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE [Signature]

. DR18-007631

INTOXILYZER 8000

Location: YAVAPAI COUNTY SO

Serial Number: 80-006228

Core Version: 8105.48

03/02/2018

21:53:57

---

Standard Lot#: 06717100A5

Last Changed By: S. BLAKLEY SR #L6861

---

QAS: S. BLAKLEY SR #L6861  
YAVAPAI CSO

---

Operator: S. BLAKLEY SR #L6861  
YAVAPAI CSO

---

Subject: SQAP, TEST

DOB: 10/20/1960

Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	21:55:00
Diagnostic Test	Pass	21:55:35
Air Blank	0.000	21:56:02
0.100 Cal Check	0.099	21:56:23
Air Blank	0.000	21:56:52
Subject Test	0.000	21:57:07
Air Blank	0.000	21:57:34
Wait	ABT*	
Air Blank	0.000	21:58:10

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

DR 18-007631

INTOXILYZER 8000

Location: YAVAPAI COUNTY SO

Serial Number: 80-006228

Core Version: 8105.48

03/02/2018

21:58:42

---

Standard Lot#: 06717100A5

Last Changed By: S. BLAKLEY SR #L6861

---

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

---

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

---

Subject: SQAP, TEST

DOB: 10/20/1960

Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	21:59:22
Air Blank	RFI**	21:59:36

\*Improper Sample

\*\*RFI Detect

Not a Successfully Completed Test Sequence

DR 18-007631

INTOXILYZER 8000  
Location: YAVAPAI COUNTY SO  
Serial Number: 80-006228  
Core Version: 8105.48  
03/02/2018 22:00:18

---

Standard Lot#: 06717100A5  
Last Changed By: S. BLAKLEY SR #L6861

---

QAS: S. BLAKLEY SR #L6861  
YAVAPAI CSO

---

Operator: S. BLAKLEY SR #L6861  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	22:01:12
Diagnostic Test	Pass	22:01:47
Air Blank	0.000	22:02:15
0.100 Cal Check	0.099	22:02:35
Air Blank	0.000	22:03:05
Subject Test	0.007*	22:06:12
Air Blank	0.000	22:06:44
Air Blank	0.000	22:07:12
Subject Test	XXX**	22:07:31
Air Blank	0.000	22:08:05

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

DR 18-007631



ARIZONA DEPARTMENT OF PUBLIC SAFETY

INTOXILYZER MAINTENANCE REPORT

CR NO. I800062280004

Page 1 of 1

AGENCY Yavapai County Sheriff's Office  
Prescott, AZ 86301  
OFFICER BLAKLEY, #6861  
DATE February 21, 2018



**INSTRUMENT**

80006228

**INSTRUMENT LOCATION**

Yavapai County Sheriff's Office

**MAINTENANCE REQUESTED**

Preventative Maintenance

**MAINTENANCE PERFORMED**

***Preventative Maintenance Performed:***

Inspection, Calibration, Verification

*Erin E. Boone*

---

ERIN E. BOONE, #5823, Forensic Scientist  
Central Regional Crime Laboratory  
2102 W. Encanto Blvd., Phoenix, AZ 85009  
602-223-2281

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)  
ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scott Blakley Sr. AGENCY Yavapai County SO  
DATE 3-2-18 TIME 22:08:37  
INTOXILYZER SERIAL # 80-006228 LOCATION YCSO DUI Truck

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

ISR 18-007631

INTOXILYZER 8000  
Location: YAVAPAI COUNTY SO  
Serial Number: 80-006228  
Core Version: 8105.48  
03/02/2018 22:08:37

-----  
Standard Lot#: 06717100A5  
Last Changed By: S. BLAKLEY SR #L6861  
-----

QAS: S. BLAKLEY SR #L6861  
YAVAPAI CSO  
-----

Operator: S. BLAKLEY SR #L6861  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	22:09:07
0.100 Cal Check	0.098	22:09:27
Air Blank	0.000	22:09:57

Preventative Maintenance  
Performed -- Timer Reset