

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. ARIZBERG 252 AGENCY Prescott P.D.
DATE 08/20/14 TIME 1321
INTOXILYZER SERIAL # 80-006230 LOCATION Prescott P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .200 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select (D) for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 099 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE T. Arizberg 252

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
08/20/2014 13:21:42

Standard Lot#: 1234567890
Last Changed By: A. A #A

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	13:22:13
0.100 Cal Check	0.099	13:22:33
Air Blank	0.000	13:23:03

Preventative Maintenance
Performed -- Timer Reset



EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST T. AMAN BAZGER 252 AGENCY Prescott P.D.
DATE 08/20/14 TIME 1324
INTOXILYZER SERIAL # 80-006220 LOCATION P.P.D.

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0.014/.000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 095 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS *

ALCOHOL HAND SANITIZER ON HANDS/WIPED LIPS & UNMOUTHED
MOUTHPIECE IMMEDIATELY PRIOR TO TEST. RE-DID ALC. FIRST CHECK:
ALL OK & FUNCTIONING CORRECTLY.
ALL FUNCTIONING CORRECTLY.

SIGNATURE T. Aman Bazger 252

INTOXILYZER 8000

Location: PRESCOTT PD

Serial Number: 80-006230

Core Version: 8105.48

08/20/2014

13:24:20

Standard Lot#: 1234567890

Last Changed By: A. A

#A

QAS: T. ATTENBERGER

#252

PRESCOTT PD

Operator: T. ATTENBERGER

#252

PRESCOTT PD

Subject: SQAP,TEST

DOB: 01/01/1900

Sex: M

Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:25:12
Diagnostic Test	Pass	13:25:45
Air Blank	0.000	13:26:13
0.100 Cal Check	0.097	13:26:34
Air Blank	0.000	13:27:03
* Subject Test	0.014	13:27:41
Air Blank	0.000	13:28:10
Wait	ABT*	
Air Blank	0.000	13:28:49

*Sequence Aborted



Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
08/20/2014

13:29:31

Standard Lot#: 1234567890
Last Changed By: A. A #A

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP, TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:30:28
Diagnostic Test	Pass	13:31:02
Air Blank	0.000	13:31:29
0.100 Cal Check	0.095	13:31:50
Air Blank	0.000	13:32:19
Subject Test	0.000	13:32:43
Air Blank	0.000	13:33:11
Wait	ABT*	
Air Blank	0.000	13:33:53

*Sequence Aborted



Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
08/20/2014 13:34:32

Standard Lot#: 1234567890
Last Changed By: A. A #A

QAS: T. ATTENBERGER #252
PRESCOTT PD

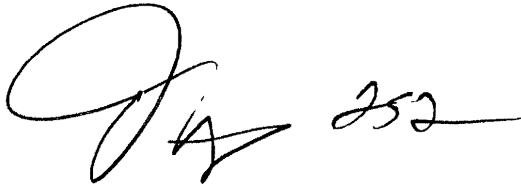
Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP, TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	13:35:31
Air Blank	0.000	13:36:00

*Improper Sample

A handwritten signature in black ink, appearing to read "T. Attenberger" followed by the number "252". The signature is stylized and cursive.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT PD

Serial Number: 80-006230

Core Version: 8105.48

08/20/2014

13:36:17

Standard Lot#: 1234567890

Last Changed By: A. A

#A

QAS: T. ATTENBERGER

#252

PRESCOTT PD

Operator: T. ATTENBERGER

#252

PRESCOTT PD

Subject: SQAP, TEST

DOB: 01/01/1900

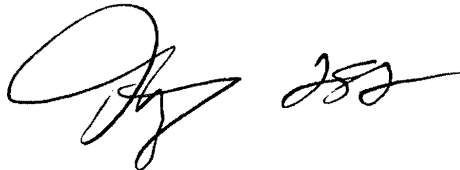
Sex: M

Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	13:37:02
Air Blank	0.000	13:37:30

*RFI Detect

Handwritten signature and initials in black ink, appearing to be 'T. Attenberger' and '252'.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
08/20/2014 13:38:07

Standard Lot#: 1234567890
Last Changed By: A. A #A

QAS: T. ATTENBERGER #252
PRESCOTT PD

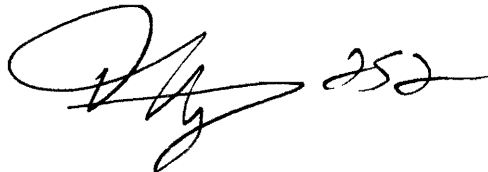
Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP, TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:38:56
Diagnostic Test	Pass	13:39:30
Air Blank	0.000	13:39:58
0.100 Cal Check	0.096	13:40:18
Air Blank	0.000	13:40:48
Subject Test	0.052*	13:43:55
Air Blank	0.000	13:44:26
Air Blank	0.000	13:44:54
Subject Test	XXX**	13:45:24
Air Blank	0.000	13:46:10

*Deficient Sample
**Invalid Sample
Begin new deprivation period



Not a Successfully Completed Test Sequence

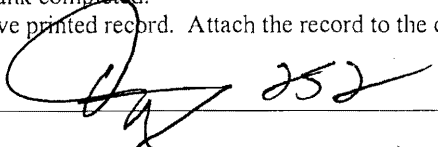
EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. ARAMBERGER 252 AGENCY Prescott PUD
DATE 08/20/14 TIME 1347
INTOXILYZER SERIAL # 80-006230 LOCATION P.P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution - LOD AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 095 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  252

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
08/20/2014 13:47:18

Standard Lot#: 1234567890
Last Changed By: A. A #A

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	13:47:50
0.100 Cal Check	0.095	13:48:11
Air Blank	0.000	13:48:40

Preventative Maintenance
Performed -- Timer Reset

A handwritten signature in black ink, appearing to be 'T. Attenberger', is written over the printed text.