

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. Arizana 252 AGENCY Prison P.D.
DATE 10/16/14 TIME 2328
INTOXILYZER SERIAL # 80-006230 LOCATION Prison P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution. .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select (D) for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 097 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
10/16/2014 23:28:33

Standard Lot#: 1234567890
Last Changed By: A. A #A

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	23:29:09
0.100 Cal Check	0.097	23:29:30
Air Blank	0.000	23:29:59

Preventative Maintenance
Performed -- Timer Reset



EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. Anderson 252 AGENCY Prescott PD
DATE 10/16/14 TIME 2330
INTOXILYZER SERIAL # 80-006230 LOCATION F.P.D.

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

DID NOT BLOW LONG ENOUGH ON "PROPER SAMPLE
RECOGNITION - MOUTH ALCOHOL". RE-DID THIS PORTION.
ALL FUNCTIONING CORRECTLY.

SIGNATURE J. Anderson 252

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
10/16/2014

23:30:43

Standard Lot#: 1234567890
Last Changed By: A. A

#A

QAS: T. ATTENBERGER
PRESCOTT PD

#252

Operator: T. ATTENBERGER
PRESCOTT PD

#252


Subject: SQAP, TEST
DOB: 01/01/1900
Sex: M

Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:31:35
Diagnostic Test	Pass	23:32:10
Air Blank	0.000	23:32:37
0.100 Cal Check	0.099	23:32:58
Air Blank	0.000	23:33:27
Subject Tes..	0.000	23:33:51
Air Blank	0.000	23:34:18
Wait	ABT*	
Air Blank	0.000	23:34:53

*Sequence Aborted

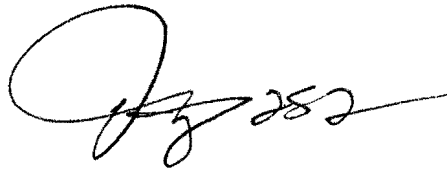

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
10/16/2014 23:36:29

Standard Lot#: 1234567890
Last Changed By: A. A #A
QAS: T. ATTENBERGER #252
PRESCOTT PD


Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP, TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	23:37:06
Air Blank	0.000	23:37:34

*Improper Sample

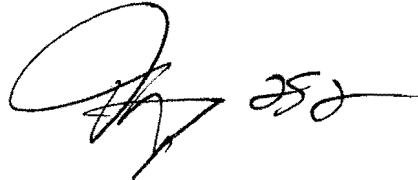A handwritten signature in black ink, appearing to be 'T. Attenberger', with the number '252' written to the right of the signature.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
10/16/2014 23:37:56

Standard Lot#: 1234567890
Last Changed By: A. A #A

QAS: T. ATTENBERGER #252
PRESCOTT PD


Operator: T. ATTENBERGER #252
PRESCOTT PD

Subject: SQAP, TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	23:38:53
Air Blank	0.000	23:39:21

*RFI Detect

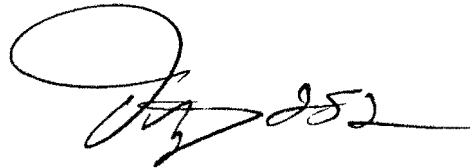

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
10/16/2014 23:40:18

Standard Lot#: 1234567890
Last Changed By: A. A #A

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD


Subject: SQAP,TEST
DOB: 01/01/1900
Sex: M Weight: 300

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:41:11
Diagnostic Test	Pass	23:41:46
Air Blank	0.000	23:42:14
0.100 Cal Check	0.099	23:42:34
Air Blank	0.000	23:43:04
Subject Test	0.040*	23:46:11
Air Blank	0.000	23:46:42
Air Blank	0.000	23:47:10
Subject Test	0.017	23:48:02
Air Blank	0.000	23:48:34
Wait		
Air Blank	0.000	23:53:29
Subject Test	XXX**	23:54:04
Air Blank	0.000	23:54:37

SEE NOTES

*Deficient Sample
**Invalid Sample
Begin new deprivation period

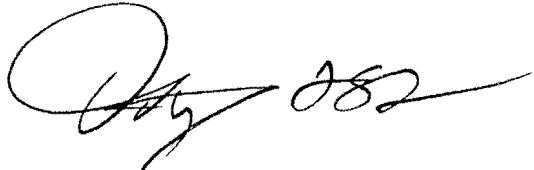

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. M. [Signature] AGENCY Phoenix PD
DATE 10/16/14 TIME 2356
INTOXILYZER SERIAL # 80-006230 LOCATION P.P.D.

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
 3. Go to the "Control Testing Menu". Select (D) for dry control test or "W" for wet control test. After selection is made press ENTER.
 4. Air blank completed.
 5. Calibration check completed. Test results 0. 099 AC.
 6. Air blank completed.
 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
10/16/2014 23:56:10

Standard Lot#: 1234567890
Last Changed By: A. A #A

QAS: T. ATTENBERGER #252
PRESCOTT PD

Operator: T. ATTENBERGER #252
PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	23:56:41
0.100 Cal Check	0.099	23:57:02
Air Blank	0.000	23:57:32

Preventative Maintenance
Performed -- Timer Reset

A handwritten signature in black ink, appearing to be 'T. Attenberger', written over the printed name of the operator.