

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. Arizon #252 AGENCY Prescott P.D.  
DATE 08/20/15 TIME 1606  
INTOXILYZER SERIAL # 80-006230 LOCATION P.P.D.

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 097 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE \_\_\_\_\_

T. Arizon #252

INTOXILYZER 8000  
Location: PRESCOTT PD  
Serial Number: 80-006230  
Core Version: 8105.48  
08/20/2015 16:06:23

-----  
Standard Lot#: 25913100A2  
Last Changed By: S. HILTON #372  
-----

QAS: T. ATTENBERGER #252  
PRESCOTT PD  
-----

Operator: T. ATTENBERGER #252  
PRESCOTT PD  
-----

Test	g/210L	Time
Air Blank	0.000	16:07:00
0.100 Cal Check	0.097	16:07:20
Air Blank	0.000	16:07:50

Preventative Maintenance  
Performed -- Timer Reset



EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST T. ANAVALGUA #252 AGENCY Prescott P.D.  
DATE 08/20/15 TIME 1609  
INTOXILYZER SERIAL # 80-006230 LOCATION P.P.D.

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.  
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.  
() 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
() 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 0916 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

ALL FUNCTIONING CORRECTLY.

SIGNATURE T. Anavalue

INTOXILYZER 8000  
Location: PRESCOTT PD  
Serial Number: 80-006230  
Core Version: 8105.48  
08/20/2015 16:09:15

---

Standard Lot#: 25913100A2  
Last Changed By: S. HILTON #372

---

QAS: T. ATTENBERGER #252  
PRESCOTT PD

---

Operator: T. ATTENBERGER #252  
PRESCOTT PD

---

Subject: SQAP, TEST  
DOB: 01/01/1900  
Sex: M Weight: 300


---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	16:10:16
Diagnostic Test	Pass	16:10:50
Air Blank	0.000	16:11:17
0.100 Cal Check	0.096	16:11:38
Air Blank	0.000	16:12:08
Subject Test	0.000	16:12:32
Air Blank	0.000	16:13:00
Wait	ABT*	
Air Blank	0.000	16:13:35

\*Sequence Aborted

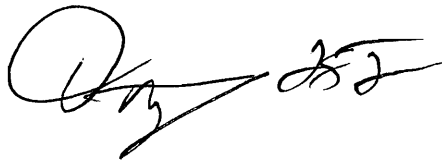

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT PD  
Serial Number: 80-006230  
Core Version: 8105.48  
08/20/2015 16:14:23

---

Standard Lot#: 25913100A2  
Last Changed By: S. HILTON #372

---

QAS: T. ATTENBERGER #252  
PRESCOTT PD

---

Operator: T. ATTENBERGER #252  
PRESCOTT PD

---

Subject: SQAP, TEST  
DOB: 01/01/1900  
Sex: M Weight: 300

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	16:15:07
Air Blank	0.000	16:15:36

\*Improper Sample



**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: PRESCOTT PD  
Serial Number: 80-006230  
Core Version: 8105.48  
08/20/2015 16:16:02

---

Standard Lot#: 25913100A2  
Last Changed By: S. HILTON #372

---

QAS: T. ATTENBERGER #252  
PRESCOTT PD

---

Operator: T. ATTENBERGER #252  
PRESCOTT PD

---

Subject: SQAP, TEST  
DOB: 01/01/1900  
Sex: M Weight: 300

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	16:16:56
Air Blank	0.000	16:17:24

\*RFI Detect



**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: PRESCOTT PD

Serial Number: 80-006230

Core Version: 8105.48

08/20/2015

16:17:46

---

Standard Lot#: 25913100A2

Last Changed By: S. HILTON

#372

---

QAS: T. ATTENBERGER

#252

PRESCOTT PD

---

Operator: T. ATTENBERGER

#252

PRESCOTT PD

---

Subject: SQAP, TEST

DOB: 01/01/1900

Sex: M

Weight: 300

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	16:18:38
Diagnostic Test	Pass	16:19:12
Air Blank	0.000	16:19:39
0.100 Cal Check	0.096	16:20:00
Air Blank	0.000	16:20:29
Subject Test	0.040*	16:23:37
Air Blank	0.000	16:24:08
Air Blank	0.000	16:24:35
Subject Test	XXX**	16:25:17
Air Blank	0.000	16:25:49

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period



**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST T. Arzoo AGENCY Prison P.D.  
DATE 08/20/15 TIME 1626  
INTOXILYZER SERIAL # 80-006230 LOCATION P.P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select D for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.096 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE T. Arzoo

INTOXILYZER 8000

Location: PRESCOTT PD

Serial Number: 80-006230

Core Version: 8105.48

08/20/2015

16:26:30

-----  
Standard Lot#: 25913100A2

Last Changed By: S. HILTON #372  
-----

QAS: T. ATTENBERGER #252

PRESCOTT PD  
-----

Operator: T. ATTENBERGER #252

PRESCOTT PD  
-----

Test	g/210L	Time
Air Blank	0.000	16:27:00
0.100 Cal Check	0.096	16:27:21
Air Blank	0.000	16:27:50

Preventative Maintenance  
Performed -- Timer Reset

