

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST S. Heenan AGENCY Prescott P.D.
DATE 11/12/17 TIME 1103
INTOXILYZER SERIAL # 80-006230 LOCATION 222 S. MYAZONA ST.

- (S) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution AC into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (S) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (S) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (S) 4. Air blank completed.
- (S) 5. Calibration check completed. Test results 0. 100 AC.
- (S) 6. Air blank completed.
- (S) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE S. Heenan

Prescott PD CHANGE 5-12-06

INTOXILYZER 8000

Location: PRESCOTT PD

Serial Number: 80-006230

Core Version: 8105.48

11/12/2017

11:03:49

Standard Lot#: 27016100A1

Last Changed By: S. HILTON #372

QAS: S. HILTON #372

PRESCOTT PD

Operator: S. HILTON #372

PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	11:04:22
0.100 Cal Check	0.100	11:04:43
Air Blank	0.000	11:05:13

Preventative Maintenance
Performed -- Timer Reset



EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST S. HERRON AGENCY PRESCOTT P.D.
DATE 11/12/17 TIME 1107
INTOXILYZER SERIAL # 80-006230 LOCATION 222 S. MARSH ST.

(S) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (S) 1. Clock time check.
(S) 2. Date check.

OPERATIONAL TESTS

- (S) 1. Alcohol-free subject test result 0. 000 AC.
(S) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(S) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(S) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS FUNCTIONING PROPERLY

0230 PLACED INTO SERVICE AS

337 SCHEDULED FOR ANNUAL DPS MAINTENANCE

SIGNATURE [Signature]

PRESCOTT PD CHANGE 5-18-06
SD NORTON #127

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
11/12/2017

11:07:10

Standard Lot#: 27016100A1
Last Changed By: S. HILTON #372

QAS: S. HILTON #372
PRESCOTT PD

Operator: S. HILTON #372
PRESCOTT PD

Subject: SQAP, TEST
DOB: 12/19/1973
Sex: M Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:08:06
Diagnostic Test	Pass	11:08:41
Air Blank	0.000	11:09:08
0.100 Cal Check	0.100	11:09:29
Air Blank	0.000	11:09:58
Subject Test	0.000	11:10:17
Air Blank	ABT*	11:10:44
Air Blank	0.000	11:11:12

*Sequence Aborted



Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT PD

Serial Number: 80-006230

Core Version: 8105.48

11/12/2017

11:11:43

Standard Lot#: 27016100A1

Last Changed By: S. HILTON

#372

QAS: S. HILTON

#372

PRESCOTT PD

Operator: S. HILTON

#372

PRESCOTT PD

Subject: SQAP, TEST

DOB: 12/19/1973

Sex: M

Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	11:12:26
Air Blank	0.000	11:12:54

*Improper Sample



Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: PRESCOTT PD

Serial Number: 80-006230

Core Version: 8105.48

11/12/2017

11:13:23

Standard Lot#: 27016100A1

Last Changed By: S. HILTON

#372

QAS: S. HILTON

#372

PRESCOTT PD

Operator: S. HILTON

#372

PRESCOTT PD

Subject: SQAP, TEST

DOB: 12/19/1973


Sex: M

Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	11:14:04
Air Blank	0.000	11:14:32

*RFI Detect

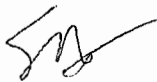

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: PRESCOTT PD
Serial Number: 80-006230
Core Version: 8105.48
11/12/2017 11:15:00

Standard Lot#: 27016100A1
Last Changed By: S. HILTON #372

QAS: S. HILTON #372
PRESCOTT PD

Operator: S. HILTON #372
PRESCOTT PD

Subject: SQAP, TEST
DOB: 12/19/1973
Sex: M Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:15:55
Diagnostic Test	Pass	11:16:30
Air Blank	0.000	11:16:58
0.100 Cal Check	0.099	11:17:19
Air Blank	0.000	11:17:48
Subject Test	0.315*	11:20:55
Air Blank	0.000	11:21:36
Air Blank	0.000	11:22:03
Subject Test	XXX**	11:22:26
Air Blank	0.000	11:23:00

*Deficient Sample

**Invalid Sample

Begin new deprivation period



Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST S. Helton AGENCY Prescott P.D.
DATE 11/12/17 TIME 1123
INTOXILYZER SERIAL # 80-006230 LOCATION 222 S. Main St.

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .140 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 099 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE S. Helton

Prescott PD CHANGE 5.12.06

INTOXILYZER 8000

Location: PRESCOTT PD

Serial Number: 80-006230

Core Version: 8105.48

11/12/2017

11:23:39

Standard Lot#: 27016100A1

Last Changed By: S. HILTON #372

QAS: S. HILTON #372

PRESCOTT PD

Operator: S. HILTON #372

PRESCOTT PD

Test	g/210L	Time
Air Blank	0.000	11:24:11
0.100 Cal Check	0.099	11:24:31
Air Blank	0.000	11:25:01

Preventative Maintenance
Performed -- Timer Reset

