

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RICHARD J. BYRNE III AGENCY AZ DPS

DATE 05/09/09 TIME 2150

INTOXILYZER SERIAL # 80-000920 DPS LOCATION WICKEN BUC

- (H) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution N/A AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (H) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (H) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (H) 4. Air blank completed.
- (H) 5. Calibration check completed. Test results 0.100 AC.
- (H) 6. Air blank completed.
- (H) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Richard J. Byrne III #6386

INTOXILYZER 8000

Location: WICKENBURG PD

Serial Number: 80-000929

05/09/2009

21:50:57

Standard Lot#: 735501G

Last Changed By: R. BYRNE, III #6386

QAS: R. BYRNE, III #6386

AZDPS

Test	g/210L	Time
Air Blank	0.000	21:51:25
Cal Check	0.100	21:51:43
Air Blank	0.000	21:52:13

Richard J. Byrne III #6386

Operator's Signature