

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RICHARD J. BYRNE III AGENCY AZ DPS

DATE 12/20/10 TIME 1323

INTOXILYZER SERIAL # 80-000929 DPS LOCATION WICKENBURG

- (PAB) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution N/A AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (PAB) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (PAB) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (PAB) 4. Air blank completed.
- (PAB) 5. Calibration check completed. Test results 0. 100 AC.
- (PAB) 6. Air blank completed.
- (PAB) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Richard J. Byrne III #6380

INTOXILYZER 8000

Location: WICKENBURG PD

Serial Number: 80-000929

Core Version: 8105.48

12/20/2010

13:23:25

Standard Lot#: 590854

Last Changed By: R. BYRNE, III #6386

QAS: R. BYRNE, III #6386

AZDPS

Operator: R. BYRNE, III #6386

AZDPS

Test g/210L Time

Air Blank 0.000 13:23:57
0.100 Cal Check 0.100 13:24:18
Air Blank 0.000 13:24:47

Preventative Maintenance
Performed -- Timer Reset