

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RICHARD J. BYRNE III AGENCY AZ DPS  
DATE 10/02/2012 TIME 0841  
INTOXILYZER SERIAL # 80-000929 LOCATION WICKENBURG P.D.

- (AB) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution N/A AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (AB) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (AB) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (AB) 4. Air blank completed.
- (AB) 5. Calibration check completed. Test results 0.098 AC.
- (AB) 6. Air blank completed.
- (AB) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Richard J. Byrne III #6386

DPS Form Exh G-2 (Rev 05-01)

STANDARD CHANGED

NEW LOT # 15712100A1

EXP 08/01/2014

