

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RICHARD J. BYRNE III AGENCY AZ DPS
DATE 08/17/2013 TIME 1657
INTOXILYZER SERIAL # 80000929 LOCATION WICKENBURG P.D.

- (P) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution N/A AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.099 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Richard J. Byrne III #6386

INTOXILYZER 8000

Location: WICKENBURG PD

Serial Number: 89-066924

Core Version: 3105.46

08/17/2013

16:57:31

Standard Lot#: 15712100A1

Last Changed By: R. BYRNE III #6386

QAS: R. BYRNE III #6386

AZ DPS

Operator: R. BYRNE III #6386

AZ DPS

Test g/210L Time

Air Blank 0.000 16:58:02
0.100 Cal Check 0.999 16:58:03
Air Blank 0.000 16:58:50

Preventative Maintenance
Performed -- Timer Reset

RECEIVED

AUG 21 2013

22