

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RICHARD J. BYRNE III AGENCY AZ DPS

DATE 09/16/2013 TIME 1944

INTOXILYZER SERIAL # 80-000929 DPS LOCATION WICKENBURG P.D.

- ~~1~~) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution N/A AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- ~~2~~) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ~~3~~) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ~~4~~) 4. Air blank completed.
- ~~5~~) 5. Calibration check completed. Test results 0.098 AC.
- ~~6~~) 6. Air blank completed.
- ~~7~~) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Richard J. Byrne III #6386

