

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RICHARD J. BYRNE III AGENCY AZ DPS

DATE 12/18/2013 TIME 1628

INTOXILYZER SERIAL # 80-000929 DPS LOCATION WICKENBURG P.D.

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution N/A AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 099 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Richard J. Byrne III # 6386



EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST RICHARD J. BYRNE AGENCY AZ DPS

DATE 12/18/2013 TIME 1634

INTOXILYZER SERIAL # 80-000929 OPS LOCATION WICKENBURG P.D.

RLB 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- RLB 1. Clock time check.
RLB 2. Date check.

OPERATIONAL TESTS

- RLB 1. Alcohol-free subject test result 0. 000 AC.
RLB 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
RLB 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
RLB 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES AD NO

COMMENTS

LOT # 15712100A1 EXP 08/01/2014

PLACE INTOX BACK IN SERVICE

SIGNATURE

Richard J. Byrne #6386



INTOXILYZER 8000

Location: WICKENBURG PD

Serial Number: 80-000929

Core Version: 8105.48

12/18/2013

16:34:01

Standard Lot#: 15712100A1

Last Changed By: R. BYRNE III #6386

QAS: R. BYRNE III #6386

AZ DPS

Operator: R. BYRNE III #6386

AZ DPS

Subject: SQAP,TEST

DOB: 08/08/1968

Sex: M

Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:35:00
Diagnostic Test	Pass	16:35:35
Air Blank	0.000	16:36:02
0.100 Cal Check	0.098	16:36:23
Air Blank	0.000	16:36:53
Subject Test	0.000	16:37:07
Air Blank	0.000	16:37:35
Wait	ABT*	
Air Blank	0.000	16:38:10

*Sequence Aborted

Not a Successfully Completed Test Sequence



INTOXILYZER 8000
Location: WICKENBURG PD
Serial Number: 80-000929
Core Version: 8105.48
12/18/2013

16:38:13

Standard Lot#: 15712100A1
Last Changed By: R. BYRNE III #6386

QAS: R. BYRNE III #6386
AZ DPS

Operator: R. BYRNE III #6386
AZ DPS

Subject: SQAP,TEST
DOB: 08/08/1968
Sex: M Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	16:38:47
Air Blank	RFI**	16:38:54

*Improper Sample
**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: WICKENBURG PD
Serial Number: 80-000929
Core Version: 8105.48
12/18/2013

16:39:23

Standard Lot#: 15712100A1
Last Changed By: R. BYRNE III #6386

QAS: R. BYRNE III #6386
AZ DPS

Operator: R. BYRNE III #6386
AZ DPS

Subject: SQAP,TEST
DOB: 08/08/1968
Sex: M Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:40:27
Diagnostic Test	Pass	16:41:02
Air Blank	0.000	16:41:30
0.100 Cal Check	0.098	16:41:51
Air Blank	0.000	16:42:20
Subject Test	0.366*	16:45:27
Air Blank	0.000	16:45:59
Air Blank	0.000	16:46:27
Subject Test	XXX**	16:46:51
Air Blank	0.000	16:47:24

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence



EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RICHARD J. BYRNE III AGENCY A2 DPS

DATE 12/18/2013 TIME 1647

INTOXILYZER SERIAL # 80-200929 DPS LOCATION WICKENBURG P.D.

- RJB* 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
Pour a standard alcohol concentration solution OR N/A AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- RJB* 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- RJB* 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- RJB* 4. Air blank completed.
- RJB* 5. Calibration check completed. Test results 0.092 AC.
- RJB* 6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *Richard J. Byrne III* # 6326

INTOXILYZER 8000

Location: WICKENBURG PD

Serial Number: 80-000929

Core Version: 8105.48

12/18/2013

16:47:36

Standard Lot#: 15712100A1

Last Changed By: R. BYRNE III #6386

QAS: R. BYRNE III #6386

AZ DPS

Operator: R. BYRNE III #6386

AZ DPS

Test	g/210L	Time
Air Blank	0.000	16:48:07
0.100 Cal Check	0.098	16:48:27
Air Blank	0.000	16:48:57

Preventative Maintenance
Performed -- Timer Reset



ARIZONA DEPARTMENT OF PUBLIC SAFETY

INTOXILYZER MAINTENANCE REPORT

AGENCY Wickenburg Police Department
Wickenburg, AZ 85390
OFFICER BYRNE III
DATE December 17, 2013

Page 1 of 1

INSTRUMENT:

80000929

MAINTENANCE REQUESTED:

Repair and Preventative Maintenance

MAINTENANCE PERFORMED:

Parts Replaced:

Breath Hose, Cal Port Check Valve, Power Supply

Preventative Maintenance Performed:

Inspection, Calibration, Verification, Adjustment

A handwritten signature in black ink, appearing to read "Joseph M Slowinski".

JOSEPH M SLOWINSKI, #6101
Electronics Technician
Central Regional Crime Laboratory
P. O. Box 6638
Phoenix, AZ 85005
(602) 223-2394



RECEIVED

JAN 6 2014

TC