

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
01/10/2008

09:08:36

Standard Lot#: 7179011
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Test	g/210L	Time
Air Blank	0.000	09:09:04
Cal Check	0.098	09:09:22
Air Blank	0.000	09:09:52

ORC

 592

Operator's Signature

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST SCARIM, ROGER G. AGENCY COTTONWOOD PD
DATE 1/10/08 TIME 0905
INTOXILYZER SERIAL # 80-000 933 LOCATION COTTONWOOD PD

- (M) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (M) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(M) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(M) 4. Air blank completed.
(M) 5. Calibration check completed. Test results 0.098 AC.
(M) 6. Air blank completed.
(M) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE OFI [Signature] 592