

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST SCARIM, ROGER G. AGENCY COTTONWOOD PD
DATE 2/27/13 TIME 0843
INTOXILYZER SERIAL # 80-000 933 LOCATION COTTONWOOD PD

(A) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (A) 1. Clock time check.
(A) 2. Date check.

OPERATIONAL TESTS

- (A) 1. Alcohol-free subject test result 0. 000 AC.
(A) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(A) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(A) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES 14 NO _____

COMMENTS

STANDARD REPLACED WITH BOTTLE LOT #
1371110041 7/1/13. DURING PROPER SAMPLE RECOGNITION
TEST INSTRUMENT RFI'D DUE TO CELL PHONE. THE
TEST WAS REPEATED.

SIGNATURE OFF [Signature] 542

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST SCARIM, ROGER G AGENCY COTTONWOOD PD
DATE 2/27/13 TIME 0914
INTOXILYZER SERIAL # 80-000933 LOCATION COTTONWOOD PD

- (1) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.106 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (1) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (1) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (1) 4. Air blank completed.
- (1) 5. Calibration check completed. Test results 0.097 AC.
- (1) 6. Air blank completed.
- (1) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE OFF [Signature] 592

DPS Form Exh G-2 (Rev 05-01)

13-002442

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
02/27/2013

08:47:54

Standard Lot#: 13711100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP, TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:49:21
Diagnostic Test	Pass	08:49:55
Air Blank	0.000	08:50:22
0.100 Cal Check	0.097	08:50:43
Air Blank	0.000	08:51:13
Subject Test	0.000	08:51:34
Air Blank	0.000	08:52:02
Wait	ABT*	
Air Blank	0.000	08:52:36

*Sequence Aborted

Not a Successfully Completed Test Sequence

13-002442

INTOXILYZER 8000

Location: COTTONWOOD PD

Serial Number: 80-000933

Core Version: 8105.48

02/27/2013

08:47:54

Standard Lot#: 13711100A1

Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592

COTTONWOOD PD

Operator: R. SCARIM #592

COTTONWOOD PD.

Subject: SQAP,TEST

DOB: 11/11/1911

Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:49:21
Diagnostic Test	Pass	08:49:55
Air Blank	0.000	08:50:22
0.100 Cal Check	0.097	08:50:43
Air Blank	0.000	08:51:13
Subject Test	0.000	08:51:34
Air Blank	0.000	08:52:02
Wait	ABT*	
Air Blank	0.000	08:52:36

*Sequence Aborted

Not a Successfully Completed Test Sequence

13- 002442

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
02/27/2013

08:47:54

Standard Lot#: 13711100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:49:21
Diagnostic Test	Pass	08:49:55
Air Blank	0.000	08:50:22
0.100 Cal Check	0.097	08:50:43
Air Blank	0.000	08:51:13
Subject Test	0.000	08:51:34
Air Blank	0.000	08:52:02
Wait	ABT*	
Air Blank	0.000	08:52:36

*Sequence Aborted

Not a Successfully Completed Test Sequence

13-002442

INTOXILYZER 8000

Location: COTTONWOOD PD

Serial Number: 80-000933

Core Version: 8105.48

02/27/2013

08:53:13

Standard Lot#: 13711100A1

Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592

COTTONWOOD PD

Operator: R. SCARIM #592

COTTONWOOD PD

Subject: SQAP, TEST

DOB: 11/11/1911

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	08:53:59
Air Blank	0.000	08:54:28

*Improper Sample

Not a Successfully Completed Test Sequence

13-002442

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
02/27/2013

08:53:13

Standard Lot#: 13711100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	08:53:59
Air Blank	0.000	08:54:28

*Improper Sample

Not a Successfully Completed Test Sequence

13-002442

INTOXILYZER 8000

Location: COTTONWOOD PD

Serial Number: 80-000933

Core Version: 8105.48

02/27/2013

08:53:13

Standard Lot#: 13711100A1

Last Changed By: R. SCARIM

#592

QAS: R. SCARIM

#592

COTTONWOOD PD

Operator: R. SCARIM

#592

COTTONWOOD PD

Subject: SQAP, TEST

DOB: 11/11/1911

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	08:53:59
Air Blank	0.000	08:54:28

*Improper Sample

Not a Successfully Completed Test Sequence

13-002442

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
02/27/2013

08:55:01

Standard Lot#: 13711100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST

DOB: 11/11/1911

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	08:55:50
Air Blank	0.000	08:56:18

*RFI Detect

Not a Successfully Completed Test Sequence

13-002442

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
02/27/2013

08:55:01

Standard Lot#: 13711100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP, TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	08:55:50
Air Blank	0.000	08:56:18

*RFI Detect

Not a Successfully Completed Test Sequence

13-02442

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
02/27/2013

08:55:01

Standard Lot#: 13711100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP, TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	08:55:50
Air Blank	0.000	08:56:18

*RFI Detect

Not a Successfully Completed Test Sequence

13-002442

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
02/27/2013

08:57:05

Standard Lot#: 13711100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:59:07
Diagnostic Test	Pass	08:59:42
Air Blank	0.000	09:00:09
0.100 Cal Check	0.097	09:00:30
Air Blank	0.000	09:00:59
Subject Test	0.056*	09:04:07
Air Blank	RFI**	09:04:24
Air Blank	0.000	09:04:52

*Deficient Sample

**RFI Detect

Not a Successfully Completed Test Sequence

13-002442

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
02/27/2013

08:57:05

Standard Lot#: 13711100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:59:07
Diagnostic Test	Pass	08:59:42
Air Blank	0.000	09:00:09
0.100 Cal Check	0.097	09:00:30
Air Blank	0.000	09:00:59
Subject Test	0.056*	09:04:07
Air Blank	RFI**	09:04:24
Air Blank	0.000	09:04:52

*Deficient Sample
**RFI Detect

Not a Successfully Completed Test Sequence

13 - 602442

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
02/27/2013

08:57:05

Standard Lot#: 13711100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:59:07
Diagnostic Test	Pass	08:59:42
Air Blank	0.000	09:00:09
0.100 Cal Check	0.097	09:00:30
Air Blank	0.000	09:00:59
Subject Test	0.056*	09:04:07
Air Blank	RFI**	09:04:24
Air Blank	0.000	09:04:52

*Deficient Sample

**RFI Detect

Not a Successfully Completed Test Sequence

13-002442

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
02/27/2013

09:05:33

Standard Lot#: 13711100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:06:31
Diagnostic Test	Pass	09:07:05
Air Blank	0.000	09:07:32
0.100 Cal Check	0.097	09:07:53
Air Blank	0.000	09:08:23
Subject Test	0.044*	09:11:30
Air Blank	0.000	09:12:03
Air Blank	0.000	09:12:31
Subject Test	XXX**	09:12:49
Air Blank	0.000	09:13:22

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

13 - 002442

INTOXILYZER 8000

Location: COTTONWOOD PD

Serial Number: 80-000933

Core Version: 8105.48

02/27/2013

09:05:33

Standard Lot#: 13711100A1

Last Changed By: R. SCARIM

#592

QAS: R. SCARIM

#592

COTTONWOOD PD

Operator: R. SCARIM

#592

COTTONWOOD PD

Subject: SQAP,TEST

DOB: 11/11/1911

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:06:31
Diagnostic Test	Pass	09:07:05
Air Blank	0.000	09:07:32
0.100 Cal Check	0.097	09:07:53
Air Blank	0.000	09:08:23
Subject Test	0.044*	09:11:30
Air Blank	0.000	09:12:03
Air Blank	0.000	09:12:31
Subject Test	XXX**	09:12:49
Air Blank	0.000	09:13:22

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

13-002442

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
02/27/2013

09:05:33

Standard Lot#: 13711100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:06:31
Diagnostic Test	Pass	09:07:05
Air Blank	0.000	09:07:32
0.100 Cal Check	0.097	09:07:53
Air Blank	0.000	09:08:23
Subject Test	0.044*	09:11:30
Air Blank	0.000	09:12:03
Air Blank	0.000	09:12:31
Subject Test	XXX**	09:12:49
Air Blank	0.000	09:13:22

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

13-002442

INTOXILYZER 8000

Location: COTTONWOOD PD

Serial Number: 80-000933

Core Version: 8105.48

02/27/2013

09:14:31

Standard Lot#: 13711100A1

Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592

COTTONWOOD PD

Operator: R. SCARIM #592

COTTONWOOD PD

Test	g/210L	Time
Air Blank	0.000	09:15:01
0.100 Cal Check	0.097	09:15:22
Air Blank	0.000	09:15:52

Preventative Maintenance
Performed -- Timer Reset

13-002442