

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST SCARIM, ROGER G. AGENCY COTTONWOOD PD
DATE 11/27/13 TIME 0738
INTOXILYZER SERIAL # 80-000933 LOCATION COTTONWOOD PD

(A) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (A) 1. Clock time check.
(A) 2. Date check.

OPERATIONAL TESTS

- (A) 1. Alcohol-free subject test result 0. 006 AC.
(A) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(A) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(A) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES A NO

COMMENTS

DURING MOUTH ALCOHOL TEST I HAD A
SNEEZING ATTACK. THAT TEST WAS REPEATED
SUCCESSFULLY.

SIGNATURE OFF [Signature] 592

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST SCARIM, ROGER G. AGENCY COTTONWOOD PD
DATE 11/27/13 TIME 0810
INTOXILYZER SERIAL # 80-000933 LOCATION COTTONWOOD PD

- (1) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.10% AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (1) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (1) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (1) 4. Air blank completed.
- (1) 5. Calibration check completed. Test results 0. 097 AC.
- (1) 6. Air blank completed.
- (1) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE ORC [Signature] 592

DPS Form Exh G-2 (Rev 05-01)

13-015304

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
11/27/2013

07:40:35

Standard Lot#: 09113100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP, TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:41:45
Diagnostic Test	Pass	07:42:20
Air Blank	0.000	07:42:47
0.100 Cal Check	0.098	07:43:08
Air Blank	0.000	07:43:37
Subject Test	0.000	07:43:54
Air Blank	0.000	07:44:21
Wait	ABT*	
Air Blank	0.000	07:44:56

*Sequence Aborted

Not a Successfully Completed Test Sequence

17-015304

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
11/27/2013

07:40:35

Standard Lot#: 09113100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP, TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:41:45
Diagnostic Test	Pass	07:42:20
Air Blank	0.000	07:42:47
0.100 Cal Check	0.098	07:43:08
Air Blank	0.000	07:43:37
Subject Test	0.000	07:43:54
Air Blank	0.000	07:44:21
Wait	ABT*	
Air Blank	0.000	07:44:56

*Sequence Aborted

Not a Successfully Completed Test Sequence

13-015304

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
11/27/2013

07:40:35

Standard Lot#: 09113100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:41:45
Diagnostic Test	Pass	07:42:20
Air Blank	0.000	07:42:47
0.100 Cal Check	0.098	07:43:08
Air Blank	0.000	07:43:37
Subject Test	0.000	07:43:54
Air Blank	0.000	07:44:21
Wait	ABT*	
Air Blank	0.000	07:44:56

*Sequence Aborted

Not a Successfully Completed Test Sequence

13-015304

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
11/27/2013

07:50:50

Standard Lot#: 09113100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST

DOB: 11/11/1911

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	07:51:37
Air Blank	0.000	07:52:06

*Improper Sample

Not a Successfully Completed Test Sequence

13-015304

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
11/27/2013

07:50:50

Standard Lot#: 09113100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	07:51:37
Air Blank	0.000	07:52:06

*Improper Sample

Not a Successfully Completed Test Sequence

13-015304

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
11/27/2013

07:50:50

Standard Lot#: 09113100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	07:51:37
Air Blank	0.000	07:52:06

*Improper Sample

Not a Successfully Completed Test Sequence

13-015304

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
11/27/2013

07:52:40

Standard Lot#: 09113100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP, TEST
DOB: 11/11/1911

Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	07:53:22
Air Blank	0.000	07:53:50

*RFI Detect

Not a Successfully Completed Test Sequence

13-015304

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
11/27/2013

07:52:40

Standard Lot#: 09113100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP, TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	07:53:22
Air Blank	0.000	07:53:50

*RFI Detect

Not a Successfully Completed Test Sequence

13-015304

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
11/27/2013

07:52:40

Standard Lot#: 09113100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	07:53:22
Air Blank	0.000	07:53:50

*RFI Detect

Not a Successfully Completed Test Sequence

13-015 304

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
11/27/2013

07:54:29

Standard Lot#: 09113100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:55:55
Diagnostic Test	Pass	07:56:30
Air Blank	0.000	07:56:57
0.100 Cal Check	0.097	07:57:18
Air Blank	0.000	07:57:47
Subject Test	0.108*	08:00:55
Air Blank	0.000	08:01:27
Air Blank	0.000	08:01:54
Subject Test	0.014	08:02:31
Air Blank	0.000	08:03:02
Wait		
Air Blank	0.000	08:07:58
Subject Test	XXX**	08:08:17
Air Blank	0.000	08:08:51

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

13-015304

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
11/27/2013

07:54:29

Standard Lot#: 09113100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP, TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:55:55
Diagnostic Test	Pass	07:56:30
Air Blank	0.000	07:56:57
0.100 Cal Check	0.097	07:57:18
Air Blank	0.000	07:57:47
Subject Test	0.108*	08:00:55
Air Blank	0.000	08:01:27
Air Blank	0.000	08:01:54
Subject Test	0.014	08:02:31
Air Blank	0.000	08:03:02
Wait		
Air Blank	0.000	08:07:58
Subject Test	XXX**	08:08:17
Air Blank	0.000	08:08:51

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

13-015 304

INTOXILYZER 8000
Location: COTTONWOOD PD
Serial Number: 80-000933
Core Version: 8105.48
11/27/2013

07:54:29

Standard Lot#: 09113100A1
Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592
COTTONWOOD PD

Operator: R. SCARIM #592
COTTONWOOD PD

Subject: SQAP,TEST
DOB: 11/11/1911
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:55:55
Diagnostic Test	Pass	07:56:30
Air Blank	0.000	07:56:57
0.100 Cal Check	0.097	07:57:18
Air Blank	0.000	07:57:47
Subject Test	0.108*	08:00:55
Air Blank	0.000	08:01:27
Air Blank	0.000	08:01:54
Subject Test	0.014	08:02:31
Air Blank	0.000	08:03:02
Wait		
Air Blank	0.000	08:07:58
Subject Test	XXX**	08:08:17
Air Blank	0.000	08:08:51

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

13-015304

INTOXILYZER 8000

Location: COTTONWOOD PD

Serial Number: 80-000933

Core Version: 8105.48

11/27/2013

08:11:34

Standard Lot#: 09113100A1

Last Changed By: R. SCARIM #592

QAS: R. SCARIM #592

COTTONWOOD PD

Operator: R. SCARIM #592

COTTONWOOD PD

Test g/210L Time

Air Blank 0.000 08:12:07
0.100 Cal Check 0.097 08:12:28
Air Blank 0.000 08:12:57

Preventative Maintenance
Performed -- Timer Reset

13-015304