



EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST SCARIM, ROGER G. AGENCY COTTONWOOD PD

DATE 1/28/15 TIME 1845

INTOXILYZER SERIAL # 80-000933 LOCATION COTTONWOOD PD

(M) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (M) 1. Clock time check.  
(M) 2. Date check.

OPERATIONAL TESTS

- (M) 1. Alcohol-free subject test result 0.000 AC.  
(M) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(M) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(M) 4. Standard Calibration Check standard 0.100 AC. Result 0.098 AC.

Instrument is operating properly and accurately. YES M NO \_\_\_\_\_

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE OR [Signature] 592

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scann, Roger G. AGENCY COTTONWOOD PD  
DATE 1/28/15 TIME 1106  
INTOXILYZER SERIAL # 80-000933 LOCATION COTTONWOOD PD

- (1) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (M) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(M) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(M) 4. Air blank completed.  
(M) 5. Calibration check completed. Test results 0.098 AC.  
(M) 6. Air blank completed.  
(M) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE off [Signature] 592

INTOXILYZER 8000  
Location: COTTONWOOD PD  
Serial Number: 80-000933  
Core Version: 8105.48  
01/28/2015

10:49:24

---

Standard Lot#: 25414100A4  
Last Changed By: R. SCARIM #592

---

QAS: R. SCARIM #592  
COTTONWOOD PD

---

Operator: R. SCARIM #592  
COTTONWOOD PD

---

Subject: SQAP, TEST  
DOB: 11/11/1911  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	10:50:52
Diagnostic Test	Pass	10:51:25
Air Blank	0.000	10:51:53
0.100 Cal Check	0.098	10:52:14
Air Blank	0.000	10:52:43
Subject Test	0.000	10:53:06
Air Blank	0.000	10:53:33
Wait	ABT*	
Air Blank	0.000	10:54:09

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

15-001096

INTOXILYZER 8000  
Location: COTTONWOOD PD  
Serial Number: 80-000933  
Core Version: 8105.48  
01/28/2015

10:49:24

---

Standard Lot#: 25414100A4  
Last Changed By: R. SCARIM #592

---

QAS: R. SCARIM #592  
COTTONWOOD PD

---

Operator: R. SCARIM #592  
COTTONWOOD PD

---

Subject: SQAP,TEST  
DOB: 11/11/1911  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	10:50:52
Diagnostic Test	Pass	10:51:25
Air Blank	0.000	10:51:53
0.100 Cal Check	0.098	10:52:14
Air Blank	0.000	10:52:43
Subject Test	0.000	10:53:06
Air Blank	0.000	10:53:33
Wait	ABT*	
Air Blank	0.000	10:54:09

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

15-001096

INTOXILYZER 8000  
Location: COTTONWOOD PD  
Serial Number: 80-000933  
Core Version: 8105.48  
01/28/2015 10:49:24

---

Standard Lot#: 25414100A4  
Last Changed By: R. SCARIM #592

---

QAS: R. SCARIM #592  
COTTONWOOD PD

---

Operator: R. SCARIM #592  
COTTONWOOD PD

---

Subject: SQAP, TEST  
DOB: 11/11/1911  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	10:50:52
Diagnostic Test	Pass	10:51:25
Air Blank	0.000	10:51:53
0.100 Cal Check	0.098	10:52:14
Air Blank	0.000	10:52:43
Subject Test	0.000	10:53:06
Air Blank	0.000	10:53:33
Wait	ABT*	
Air Blank	0.000	10:54:09

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

15- 501096

INTOXILYZER 8000  
Location: COTTONWOOD PD  
Serial Number: 80-000933  
Core Version: 8105.48  
01/28/2015

10:54:56

---

Standard Lot#: 25414100A4  
Last Changed By: R. SCARIM #592

---

QAS: R. SCARIM #592  
COTTONWOOD PD

---

Operator: R. SCARIM #592  
COTTONWOOD PD

---

Subject: SQAP, TEST  
DOB: 11/11/1911  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	10:55:42
Air Blank	0.000	10:56:10

\*Improper Sample

**Not a Successfully Completed Test Sequence**

15-001096

INTOXILYZER 8000  
Location: COTTONWOOD PD  
Serial Number: 80-000933  
Core Version: 8105.48  
01/28/2015 10:54:56

---

Standard Lot#: 25414100A4  
Last Changed By: R. SCARIM #592

---

QAS: R. SCARIM #592  
COTTONWOOD PD

---

Operator: R. SCARIM #592  
COTTONWOOD PD

---

Subject: SQAP,TEST  
DOB: 11/11/1911  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	10:55:42
Air Blank	0.000	10:56:10

\*Improper Sample

Not a Successfully Completed Test Sequence

15-001096

INTOXILYZER 8000  
Location: COTTONWOOD PD  
Serial Number: 80-000933  
Core Version: 8105.48  
01/28/2015

10:54:56

---

Standard Lot#: 25414100A4  
Last Changed By: R. SCARIM #592

---

QAS: R. SCARIM #592  
COTTONWOOD PD

---

Operator: R. SCARIM #592  
COTTONWOOD PD

---

Subject: SQAP,TEST  
DOB: 11/11/1911  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	10:55:42
Air Blank	0.000	10:56:10

\*Improper Sample

**Not a Successfully Completed Test Sequence**

15-001096

INTOXILYZER 8000  
Location: COTTONWOOD PD  
Serial Number: 80-000933  
Core Version: 8105.48  
01/28/2015

10:56:44

---

Standard Lot#: 25414100A4  
Last Changed By: R. SCARIM #592

---

QAS: R. SCARIM #592  
COTTONWOOD PD

---

Operator: R. SCARIM #592  
COTTONWOOD PD

---

Subject: SQAP,TEST  
DOB: 11/11/1911  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	10:57:30
Air Blank	0.000	10:57:59

\*RFI Detect

**Not a Successfully Completed Test Sequence**

15-001096

INTOXILYZER 8000  
Location: COTTONWOOD PD  
Serial Number: 80-000933  
Core Version: 8105.48  
01/28/2015

10:56:44

---

Standard Lot#: 25414100A4  
Last Changed By: R. SCARIM #592

---

QAS: R. SCARIM #592  
COTTONWOOD PD

---

Operator: R. SCARIM #592  
COTTONWOOD PD

---

Subject: SQAP,TEST  
DOB: 11/11/1911  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	10:57:30
Air Blank	0.000	10:57:59

\*RFI Detect

**Not a Successfully Completed Test Sequence**

15 - 00 1096

INTOXILYZER 8000

Location: COTTONWOOD PD

Serial Number: 80-000933

Core Version: 8105.48

01/28/2015

10:56:44

---

Standard Lot#: 25414100A4

Last Changed By: R. SCARIM

#592

---

QAS: R. SCARIM

#592

COTTONWOOD PD

---

Operator: R. SCARIM

#592

COTTONWOOD PD

---

Subject: SQAP,TEST

DOB: 11/11/1911

Sex: M

Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	10:57:30
Air Blank	0.000	10:57:59

\*RFI Detect

**Not a Successfully Completed Test Sequence**

15-001096

INTOXILYZER 8000  
Location: COTTONWOOD PD  
Serial Number: 80-000933  
Core Version: 8105.48  
01/28/2015

10:58:32

---

Standard Lot#: 25414100A4  
Last Changed By: R. SCARIM #592

---

QAS: R. SCARIM #592  
COTTONWOOD PD

---

Operator: R. SCARIM #592  
COTTONWOOD PD

---

Subject: SQAP, TEST  
DOB: 11/11/1911  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	10:59:30
Diagnostic Test	Pass	11:00:04
Air Blank	0.000	11:00:32
0.100 Cal Check	0.098	11:00:53
Air Blank	0.000	11:01:22
Subject Test	0.072*	11:04:30
Air Blank	0.000	11:05:02
Air Blank	0.000	11:05:29
Subject Test	XXX**	11:05:47
Air Blank	0.000	11:06:21

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

15-001096

INTOXILYZER 8000  
Location: COTTONWOOD PD  
Serial Number: 80-000933  
Core Version: 8105.48  
01/28/2015

10:58:32

---

Standard Lot#: 25414100A4  
Last Changed By: R. SCARIM #592

---

QAS: R. SCARIM #592  
COTTONWOOD PD

---

Operator: R. SCARIM #592  
COTTONWOOD PD

---

Subject: SQAP, TEST  
DOB: 11/11/1911  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	10:59:30
Diagnostic Test	Pass	11:00:04
Air Blank	0.000	11:00:32
0.100 Cal Check	0.098	11:00:53
Air Blank	0.000	11:01:22
Subject Test	0.072*	11:04:30
Air Blank	0.000	11:05:02
Air Blank	0.000	11:05:29
Subject Test	XXX**	11:05:47
Air Blank	0.000	11:06:21

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

15-001096

INTOXILYZER 8000

Location: COTTONWOOD PD

Serial Number: 80-000933

Core Version: 8105.48

01/28/2015

11:07:50

-----  
Standard Lot#: 25414100A4

Last Changed By: R. SCARIM #592  
-----

QAS: R. SCARIM #592

COTTONWOOD PD  
-----

Operator: R. SCARIM #592

COTTONWOOD PD  
-----

Test	g/210L	Time
Air Blank	0.000	11:08:20
0.100 Cal Check	0.098	11:08:41
Air Blank	0.000	11:09:11

Preventative Maintenance  
Performed -- Timer Reset

15-001096